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Read carefully this instruction for use because this contains important information for you concerning the utilization of the medical device.

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1. *Item 40060023 biopsy valve is designed to be used with Olympus endoscopes,*
2. *Item 40060024 biopsy valve is designed to be used with Fujinon endoscopes,*
3. *Item 40060025 biopsy valve is designed to be used with Pentax endoscopes,*
4. *Our biopsy valves are designed to allow passage of a 3.2mm device and once the device is removed to seal securely.*

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CAUTION !

- When irrigation is used, ensure proper techniques to avoid aspiration in the patient.
- Do not use a sharp or pointed object to prime the biopsy valve prior to use.
- Do not leave a device hanging from the valve. Doing so can cause the creation of larger valve slit/hole that can compromise leak management.
- If the lid of the biopsy valve is opened while attached to the endoscope during a procedure, scope suction will be compromised and chances of leakage increase. Sterile gauze should be used to cover the biopsy valve if the lid must be opened for any reason.
- Exposure to bodily fluids may occur during connection or disconnection of these devices; adherence to Body Substance Isolation protocols is the responsibility of the user.
- Products are for single-use only. Do not reuse, reprocess, or resterilize single-use products. Reuse, reprocessing, or resterilization may compromise the structural integrity of the device and/or lead to device failure which in turn may result in patient injury, illness, or death. Reuse, reprocessing, or resterilization may also increase the risk of contamination of the device and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient. Fartley assumes no liability with respect of devices reused, reprocessed, or resterilized and makes no warranties, express or implied, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose, with respect to any such device.

Champ d'application / Scope of application

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The item 40060023 biopsy valve is used to cover the opening to the biopsy/suction channel inlet of Olympus Gastrointestinal endoscopes. The item 40060024 biopsy valve is used to cover the opening to the biopsy/suction channel inlet of Fujinon (G5 series and newer) Gastrointestinal endoscopes. The item 40060025 biopsy valve is for use with Pentax gastrointestinal endoscopes. These three items provide access for endoscopic device passage and exchange, helps maintain insufflation and minimizes leakage of biomaterial from the biopsy port throughout the gastrointestinal endoscopic procedure.

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Use and hygiene precautions include, but are not limited to, those specific to any endoscopic procedure.

Contre-indications / Contraindications

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Contraindications include, but are not limited to, those specific to any endoscopic procedure.

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*Prior to clinical use, inspect and familiarize yourself with the device. If there is evidence of damage, **do not use this product and contact your local Product Specialist.***

Conditions de stockage et de transport / Condition of storage and carriage

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Products should be stored in ventilated, dry and non-corrosive rooms.

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Respect the destruction rules of the establishment

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Package: 1 pcs/bag, 100 pcs/box, 400 pcs/carton

Sterilization: Sterile

Validity of the sterilization: three years

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Yangzhou Fartley Medical Instrument Technology Co.,Ltd

Beizhou Road, Lidian Town, Guangling District, Yangzhou, Jiangsu, China

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FTL-DPS-IFU-01

Biopsy valve Instruction for use- V1 of 29/11/2017

Revision : A/0

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Prolinx GmbH