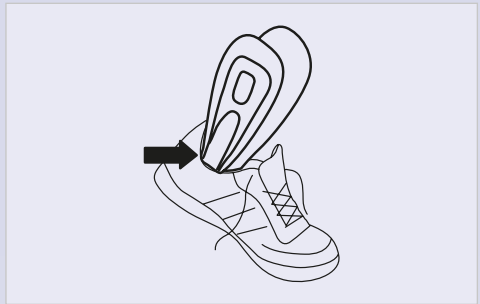
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

- Moderate to severe ankle sprains or strains; lateral ligament injuries, chronic instability

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by products.

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

1. Find the letter E, inscribed on the outer part of one of the rigid shells. Place the brace inside the shoe (left or right), making sure the letter E is placed on the exterior part of the ankle.
2. You can place the shoe sole over the brace's footplate for better comfort. This is highly recommended if wearing orthopedic insoles.
3. Place your foot inside the shoe, fitting the foam liners along the ankle, and the cavities over the malleoli, for maximum comfort and protection.
4. Make sure the lateral shells and liners fit snugly, and then close the brace using the contact closure straps.
5. Lace-up your shoes to the desired comfort. Important – The ankle brace is delivered with a set of screws. These are to be used if limited range of motion is needed; this is sometimes the case after fresh injuries. In order to block the brace hinges, place and tighten the screws in the holes provided for this purpose.

NOTE: It is recommended to the wear device over a sock

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION

70% polyamid - 20% neoprene

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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