ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE
The user should be able to:
- Read and understand the directions, warnings and cautions.

INTENDED USE/INDICATIONS
• Mild to moderate ankle sprains and strains.
• Chronic ankle instabilities.

CONTRAINDICATIONS
• None

WARNING & PRECAUTIONS
The product is intended for use on a single patient. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS
1. Place the foam liners pads inside the rigid bilateral shells.
2. Adjust the heel strap for proper fit.
3. Pull the outer shells and foam liners along the ankle, making sure the malleoli are well protected.
4. Make sure the brace fits snugly, and then close the brace using the contact closure straps.
5. If necessary, adjust the heel pad with the vertical side straps while the brace is on. Pull straps upward until desired fit is obtained.
6. For maximum comfort and support, wear a long absorbent sock and a laced athletic shoe when wearing the brace.

CARE
Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION
Polyester, polyurethane, polyamide, silicone, polypropylene

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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