
Playmaker XPert

SLEEVE

WRAP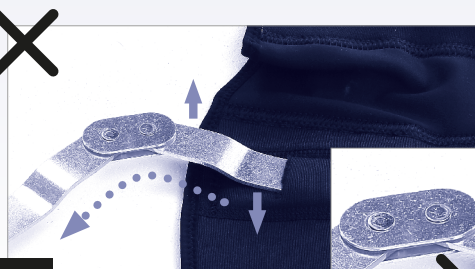

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

- The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance.
- The user should be able to: read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Support for mild MCL and/or LCL sprains
- Mild arthritis
- Mild global instabilities
- Post arthroscopic meniscectomy (Sleeve version)

CONTRAINDICATIONS:

None

WARNING & PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.
- Please be advised that DJO, LLC, does not recommend simultaneous wearing of functional knee braces bilaterally (right and left legs), during activities (ie skiing) that may cause the braces to contact each other, resulting in potential loss of control and injury. Functional knee braces are not recommended to be worn over clothing.

APPLICATION INSTRUCTIONS

1. Unfasten and loosen the straps at the thigh and calf.
 2. Position the brace so that the hinges are parallel to each other, and the top of the circular cut-out aligns with the top of the kneecap.
 3. Fasten the support straps at the thigh and calf.
4. Removable polycentric hinges

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION

Polyethersulfone, Nylon, Spandex, Elastane, Polyamide, Polyurethane, Cotton, Polyester, Plastic (Acetal & PVC), Synthetic Rubber (SBR/CR), Aluminium

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2017 DJO, LLC 13-00130 Rev A
MDS GmbH