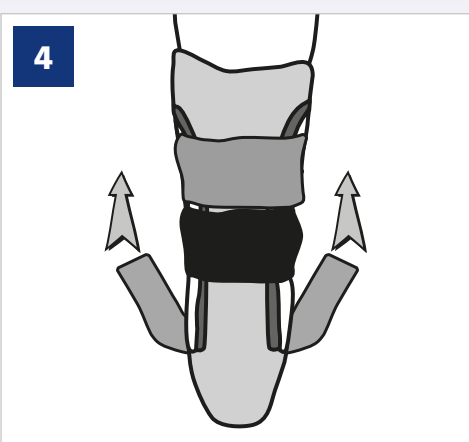
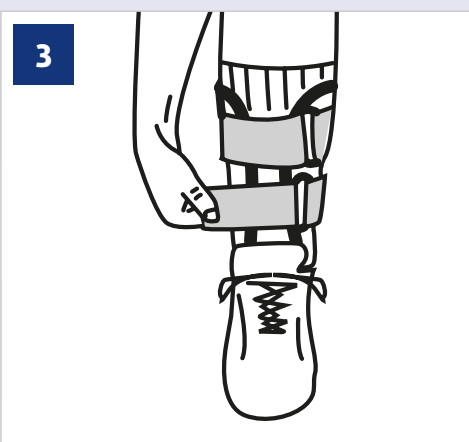
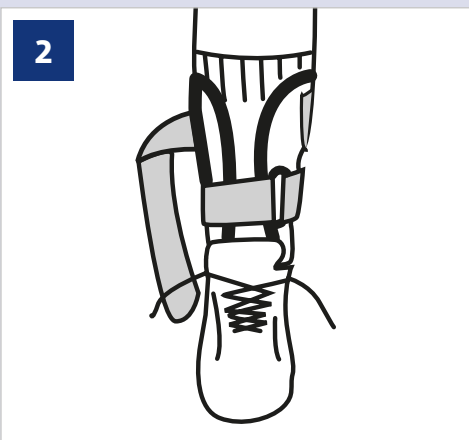
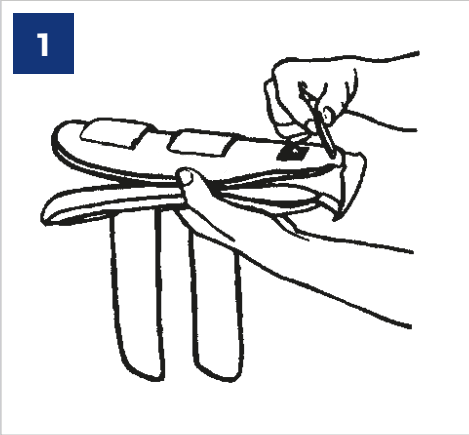
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS
• Support for acute or chronic injuries of the ankle.

NOTE: The Surround gel ankle brace should be worn over a piece of stockinette or thin cotton sock.

CONTRAINDICATIONS
• None

WARNING & PRECAUTIONS
If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INSTRUCTIONS
1) Adjust and Center Heel Pad: Use the two side pulls on both shells to center the heel pad. Adjust the heel pad as needed so brace is snug and fits comfortably.

2) Position shells on either side of ankle, align with midline of leg. Secure lower strap first for proper alignment. Step into and lace-up your shoe.

3) Apply pressure to the sides of the therapeutic gel bag to allow even flow of pressure within the plastic shell. Wrap the upper strap around calf and secure. Adjust both straps for patient comfort. Be certain the side shells are centered along the ankle and leg. Straps may be trimmed if necessary.

4) If necessary, adjust the heel pad with the vertical side straps while the brace is on. Using both hands, pinch bottom of shells, and pull straps upward until desired fit is obtained.

FOR COLD THERAPY USE
• Remove therapeutic gel bag from ankle support system and place in freezer for 2 hours.
• Freeze only therapeutic bag. DO NOT place entire system in freezer.
• Place gel bag in ankle support and follow fitting instructions.
• NOTE: absorbent sock should be worn before ankle support is fitted.

CARE
Hand wash in cold water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

LATEX NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STRIVING FOR THE HIGHEST LEVEL OF COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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