
1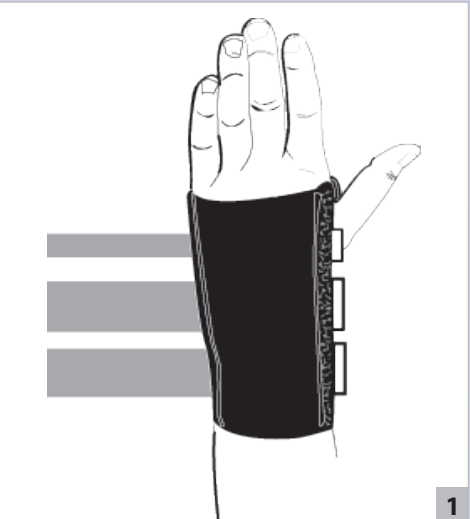

2

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

- Support for Carpal Tunnel Syndrome, immobilization of the wrist.

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

- 1) Place hand in splint with thumb in side slot. The stay should be resting on the palm and in line with the arm.
- 2) Wrap and secure closures. Feed straps through D-rings and secure.

The support should be snug but comfortable.

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CARE

Hand wash in cold water using mild soap, rinse thoroughly. **AIRDRY.**
Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

NOT MADE WITH NATURAL RUBBER LATEX.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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