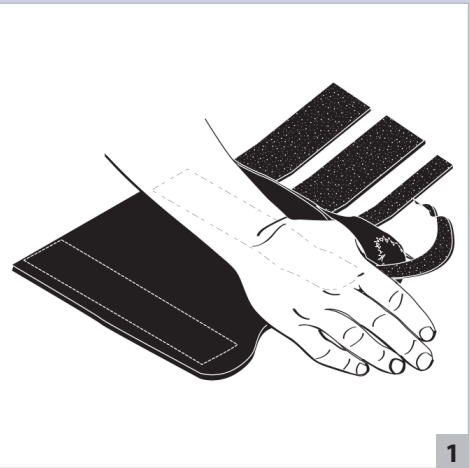
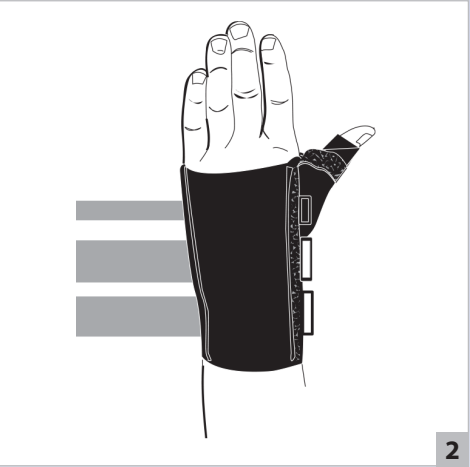
1

2

3

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

- Immobilization of wrist and thumb due to tendonitis, Carpal Tunnel Syndrome (CTS) or deQuervain's syndrome.

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

- 1) Loosen straps and slip hand into support with thumb resting in abduction stay. The palmar stay should be resting on palm and wrist area of the hand.
Note: Stays are malleable and may be contoured to fit individual needs.
- 2) Wrap and secure thumb strap.
- 3) Starting at the hand, feed straps through "D" rings and secure, working up the arm.

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CARE

Hand wash in cold water using mild soap, rinse thoroughly. AIRDRY.
Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

NOT MADE WITH NATURAL RUBBER LATEX.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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