
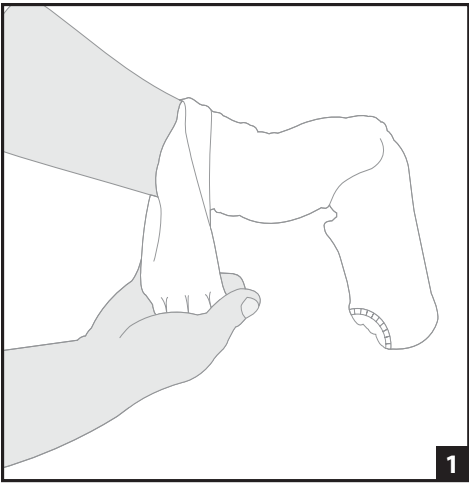
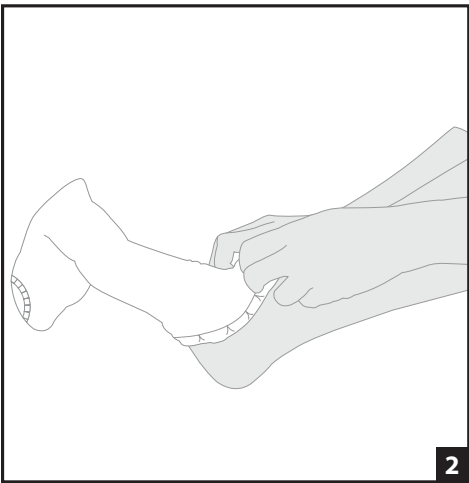
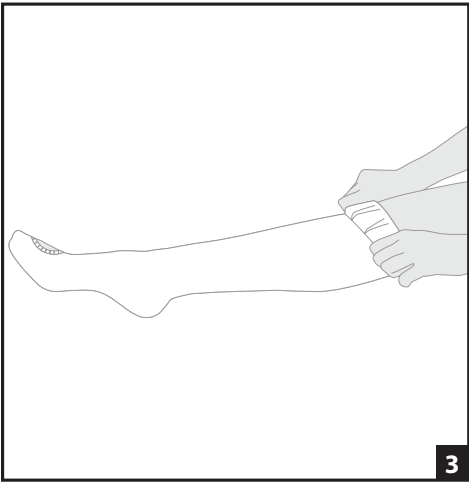
1

2

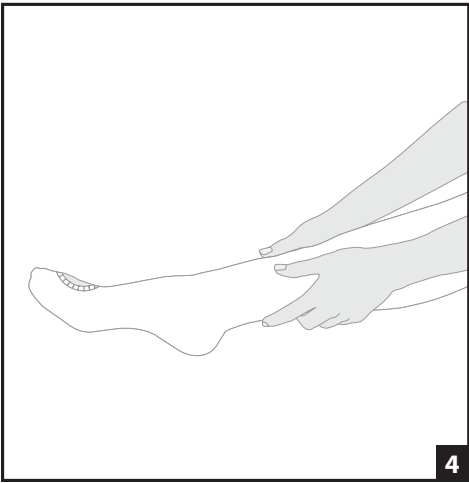3

4

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE / INDICATIONS

- Aircast VenaPure™ Anti-Embolism stockings are designed to help prevent deep vein thrombosis in medical or surgical patients.
- They provide graduated compression, applying less pressure as you move up the leg.
- This generates a pumping effect which improves circulation by accelerating the venous return, helps prevent the formation of blood clots (embolism), and also helps decrease swelling in the legs.
- Aircast VenaPure™ Anti-Embolism stockings are indicated for pre and post-operative use, for bed-ridden patients and for conditions such as venous disorders and varicose veins.

CAUTION

- Do not apply over open skin or recent incisions.
- Use with caution with patients with sensitivity of limbs due to underlying conditions (i.e. Phlebitis)

CONTRA-INDICATIONS

- Dermatitis with open wound or gangrene.
- Severe arteriosclerosis or other ischemic vascular disease or the absence of pedal pulse.
- Excessive leg edema or extreme deformity of the legs.
- Diabetic conditions with compromised circulation.

WARNING: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INSTRUCTIONS

Determine the correct stocking size by using the sizing table on the Dispenser box. Once on, do not roll down the top of the stockings, make sure they do not shift or slip.

- Holding the stocking, insert your hand as far as the heel. Grasping the heel, turn the stocking inside-out, all the way down to the heel area.
- Position the stocking over the foot and heel, make sure the heel is correctly centered.
- Pull the stocking up slowly over the ankle and calf.
 - For Thigh Length Models, pull the stocking up slowly over the ankle, calf and thigh.
 - Make sure that the gusset is placed on the inside of the thigh, covering the femoral artery.
 - For the model with Waist Attachment, secure the closure around the waist.
- Smooth out any creases. Ensure positioning at toes and heel for proper fit.

CARE INSTRUCTIONS

- Place stockings in a hosiery bag and machine wash in hot water on gentle cycle (max 70°C - 158°F).
- Tumble dry on medium heat (max 80°C - 176°F, for 15-20 minutes maximum).
- If bleaching is desired, do not use a chlorine bleach, use peroxide based bleach only.
- Do not expose to direct heat sources such as irons, blow dryers, or radiators.
- Do not dry clean.
- The Aircast VenaPure™ Anti-Embolism stockings need to be replaced after approximately 30 washes.
- Aircast VenaPure™ Anti-Embolism stockings should be replaced by a clean pair every 3 days, on average.

FOR SINGLE PATIENT USE ONLY.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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©2019 DJO LLC
AIRCAS[®]DJO Global LLC
Vista, CA 92081-5535 USAA DJO, LLC