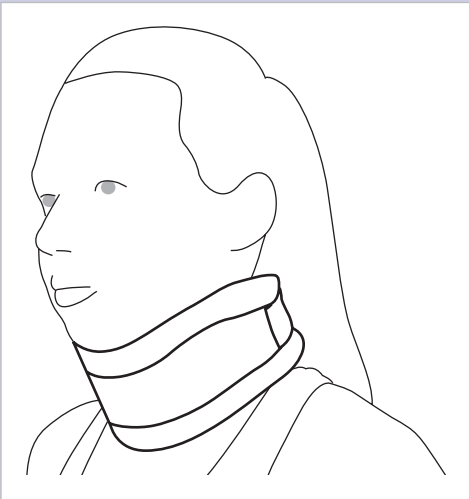
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

- Ideal for supporting the cervical spine in neutral position, to be used for cervical sprains and strains, post-injury rehabilitation after wearing rigid collar use.

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

- 1) Undo the contact closure strap to open the collar.
- 2) Place the collar around the neck, with the contoured part under the chin.
- 3) Close the collar using the contact closure strap.
- 4) Adjust the collar's tightness according to needs and comfort.

CARE

Hand-wash in cold water with mild soap and airdry. Do not bleach. Do not dry-clean or machine-dry. Do not iron.

COMPOSITION

100% cotton

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2015 DJO, LLC
MDSS GmbH