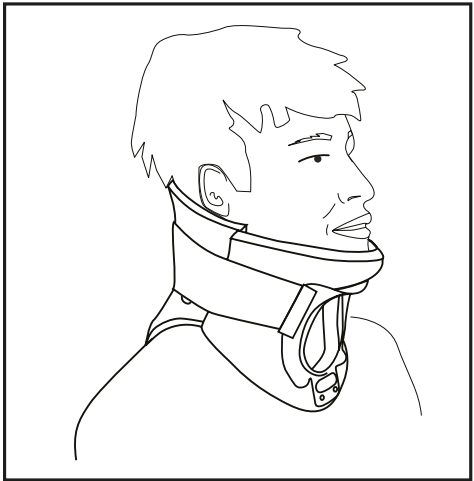
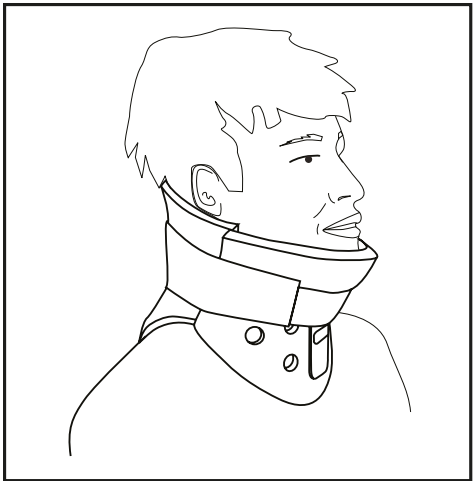
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

User should be able to:

- Read and understand the directions, warnings and cautions.

INTENDED USE/INDICATIONS: Restricts cervical spine flexion, extension and rotation to promote patient recovery.

CONTRAINDICATIONS: None.

WARNINGS AND CAUTIONS: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INFORMATION:

1. After selecting the correct size, apply the BACK piece of the collar to the back of the patient's neck. The collar should be centered and the front arrow should point upward.
2. Apply the FRONT piece of the collar to the front of the patient's neck with the chin secured in recess. Center the collar to ensure neutral alignment. The FRONT piece of the collar should overlap the BACK piece of collar.
3. To secure the collar, bring the straps from the BACK piece around to the FRONT piece. Improvement of immobilization and comfort is achieved by minor bilateral adjustments of the straps.
4. If needed, a cotton stockinet is available for use on the FRONT and BACK panel of the collar for additional patient comfort.

CLEANING INSTRUCTIONS: Hand wash in warm water using mild soap. Rinse thoroughly. AIR DRY separately.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

Rx ONLY.

INTENDED FOR SINGLE PATIENT USE.
NOT MADE WITH NATURAL RUBBER LATEX.

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