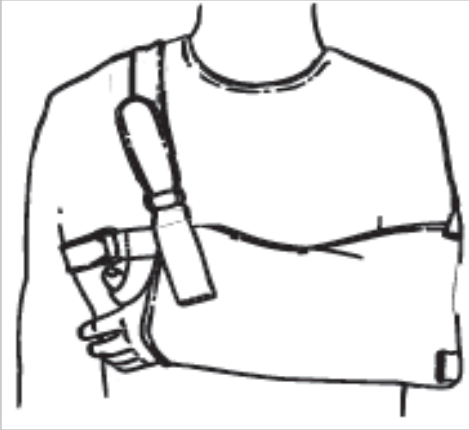
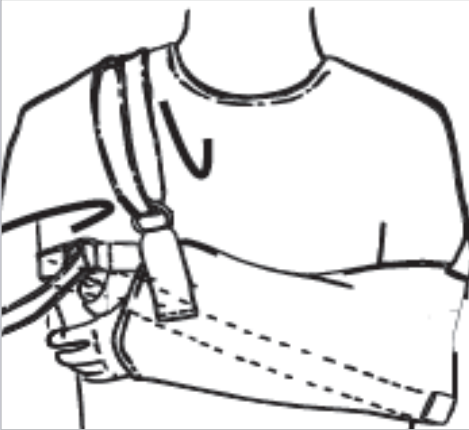
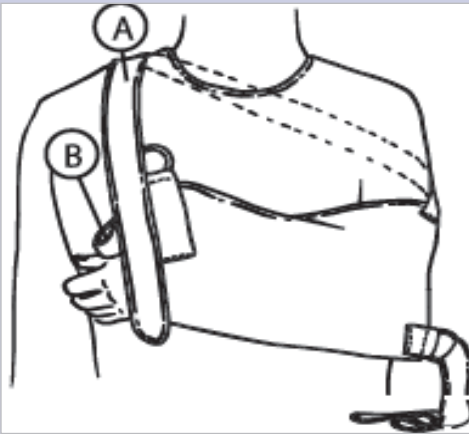
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

- Immobilisation and support in all cases requiring restricted range of motion of upper limbs (shoulder or elbow trauma).

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

Position the injured limb in the sling and the shoulder strap (A) diagonally on the opposite side of the injured limb.

- 1) Position the elbow first, then the forearm in the sling. You can position the thumb in the finger loop (B) located inside the sling, in order to keep the hand in semi-pronation.
- 2) Insert the shoulder strap through the back O-ring, adjust its length and secure using the hook and loop.
- 3) Insert the shoulder strap through the front O-ring, adjust its length and secure it using the hook & loop. We recommend requesting the help of another person to position and adjust the sling correctly.

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CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

 **LATEX FREE**

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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