
1


2


3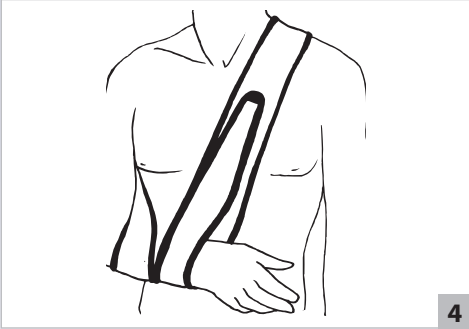

4

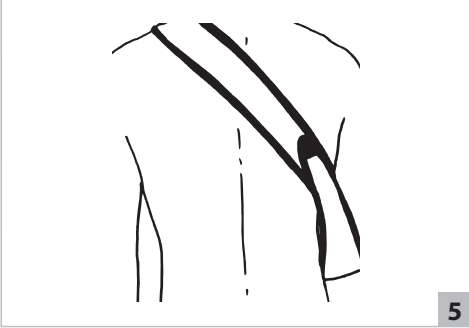5

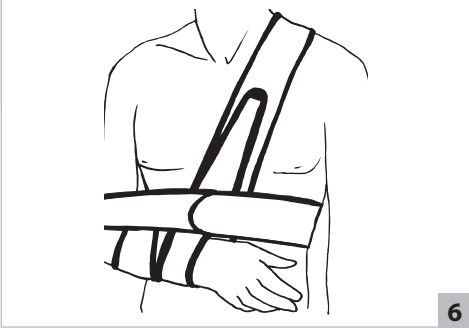6

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

- Upper extremity trauma (post-surgical repair, humeral lesions) Neurology: paralysis, hemiplegia.

CONTRAINDICATIONS

- Make sure the strap used as the arm sling is correctly positioned. Avoid placing on the spine.

WARNING & PRECAUTIONS

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

Various options: see drawings.

A WITH STRAIGHT STRAPS

- 1) Arm strap sling.
- 2) With swathe.
- 3) With arm strap sling & swathe.

B WITH CROSSED STRAP OVER THE BACK

- 4) Arm strap sling.
- 5) Back view.
- 6) With swathe.

Use the longer strip as an arm strap sling: use the contact closure tabs to make a loop at each end to hold the arm. Use the shorter strip as swathe.

CARE

Wash in cold water. Do not bleach. Do not dryclean. Do not iron.

COMPOSITION

74% Polyamide - 26% Cotton

LATEX FREE

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2015 DJO, LLC 13-0315 REV D