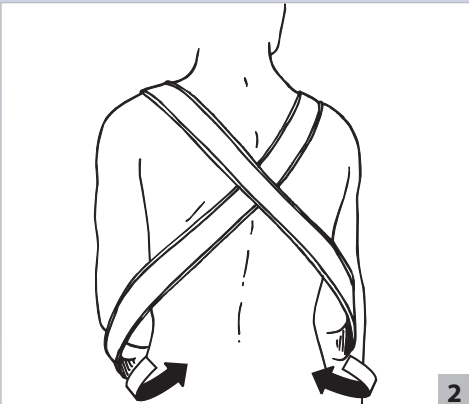
1

2

3

4

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

• Upper extremity trauma. Neurology: paralysis, hemiplegia.

CONTRAINDICATIONS

• None

WARNING & PRECAUTIONS

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

- 1) Place the forearm in the arm sling, the label facing outwards and the straps going up.
- 2) Pass the straps over the shoulders.
- 3) Cross the straps over the back.
- 4) Bring the straps back to the front, passing them under the arm sling. Attach the straps to the arm sling, using the contact closure tabs.
- 5) To immobilize the arm against the body, attach the swathe to the arm sling and wrap it around the thorax.

CARE

Wash in cold water. Do not bleach. Do not dryclean. Do not iron.

COMPOSITION

61% Polyamide - 30% Cotton - 5% Polyester - 4% Fibranne

LATEX FREE

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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COMPOSITION

61% Polyamide - 30% Coton - 5% Polyester - 4% Fibranne

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