ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE
The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Support and immobilization of upper extremity post-op / post-trauma
- Acromioclavicular sprains, dislocations
- Clavicle fractures, humeral head fractures
- Post-op use
- Neurology: paralysis, hemiplegia
- Rheumatology: shoulder arthritis (frozen shoulder)

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS
If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

- 1) Position the brace by placing the shoulder pad onto the shoulder needing immobilization. As the device is bilateral, simply reverse for use on opposite side.
- 2) Once the shoulder pad is correctly positioned, lift opposite arm up in order to position the brace correctly. Wrap the front part of the brace around the torso and the opposite arm. Then secure it to the back of the brace, using the hook & loop closure.
- 3) Wrap the forearm in the arm sling. Adjust the height of the arm sling to ensure correct position of the forearm. Close the arm sling using the hook & loop strips located at the top of the front of the brace. The Donjoy label should be facing outwards.
- 4) Fasten the posterior strap to the front of the brace in order to maintain the elbow in correct position.

CARE
Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION
Polyamide, Polyester, Polyurethane, Synthetic resin, Polypropylene, Natural rubber

LATEX CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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