
A60™ Ankle Brace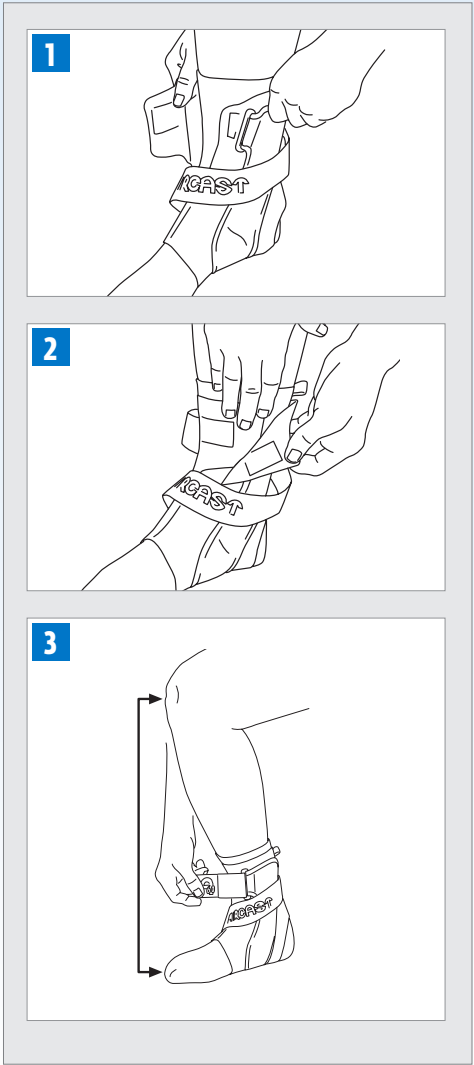

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

THE USER SHOULD BE ABLE TO:

- READ AND UNDERSTAND THE DIRECTIONS, WARNINGS AND CAUTIONS.

INTENDED USE /INDICATIONS :

Designed to provide support and protection for ankles. Remark: For maximum comfort and support, wear an absorbent sock and a laced athletic shoe when wearing the brace.

CONTRAINDICATIONS :

None

WARNINGS AND PRECAUTIONS :

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INFORMATION :

- 1). Unfasten strap from black plastic ring, open top flaps, and open brace. Slide foot into top of brace. Pull brace stabilizers upward and align stabilizers with sides of ankle.

- 2). Wrap flaps around top of ankle and secure. Insert end of strap into black ring.

- 3). Bend knee and align over toes. Pull strap until snug and secure strap end onto body of strap.

CLEANING INSTRUCTIONS :

Hand wash in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

 **NOT MADE WITH NATURAL RUBBER LATEX**

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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