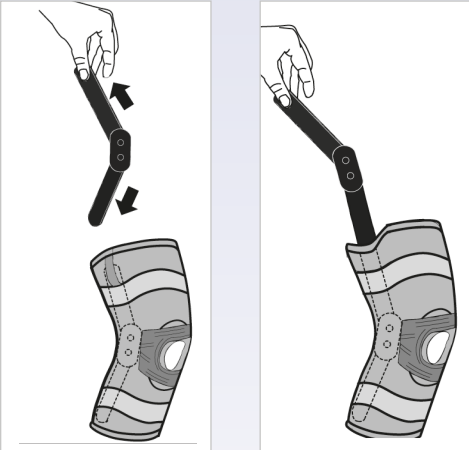
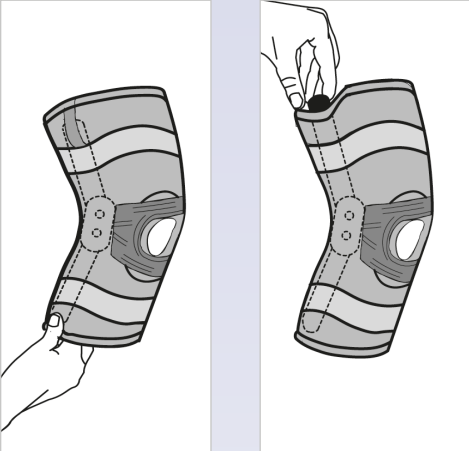
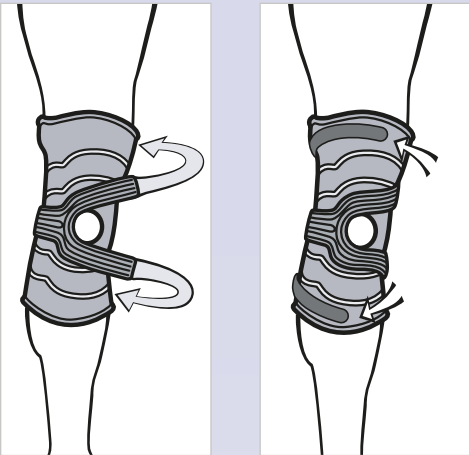
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

- For symptomatic relief of patellofemoral dysfunction including dislocation and/or subluxation, maltracking and patellar tendonitis.

CONTRAINDICATIONS

- Not for use in water sports.

WARNING & PRECAUTIONS

Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products. If you experience any pain, swelling, sensation changes, any unusual reaction while using this product, consult your medical professional.

APPLICATION INSTRUCTIONS

- 1) Loosen and release straps. Pull sleeve onto leg using finger loops. Situate the sleeve on the leg so that the patella opening surrounds the kneecap.
- 2) Each brace has two buttresses to choose from, select the most comfortable buttress. Affix the buttress on the inside of sleeve around patella opening. Properly position on the medial or lateral side of the patella opening to address indication.
- 3) Wrap the bottom strap around the calf and affix the strap end to the sleeve using the contact closure.
- 4) Wrap the top strap around the thigh and affix the strap end to the sleeve using the contact closure. Adjust the tension on the patella by pulling the tension straps tighter around the thigh and calf.
- 5) To remove hinges: Hold the lower portion of the brace by the hinge.
- 6) Pull up on material at top of hinge casing to allow the top of the hinge bar to pop out of slit in hinge cover. Remove hinge.
- 7) To insert hinges: Ensure the hinge bends in direction of knee before inserting. Open slit in hinge cover and insert hinge bar.

CARE

Remove hinges before cleaning. Hand wash sleeve in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material.

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**NOT MADE WITH NATURAL RUBBER LATEX**

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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