ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE
The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS
Immobilisation of the wrist/thumb/fingers as part of pain management

CONTRAINDICATIONS
• None

WARNING & PRECAUTIONS
• We recommend that the first application be made under supervision of a health care professional.
• If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
• If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.

APPLICATION INSTRUCTIONS
1) Place the thumb immobilization piece directly on the skin, on the thumb commissure area.
Longer part of the piece must cover the thumb.

See the pictogram  to place the piece correctly.

2) Loosen straps and slip hand into brace
3) Make sure the finger stay is the correct height (fingers should fit within the frame). Adjust the stay height if needed and ensure stay cover is properly attached to the main fabric hook.
4) Place the thumb in the corresponding opening at the top of brace (left or right).

Note:
a) The brace textile must cover the thumb immobilization piece, in order to maintain its position
b) The second opening must remain empty.
c) The hand and wrist should be gripped between the two stays (palmar and dorsal).
d) If needed, move the dorsal stay to ensure correct position.
e) Stays are malleable and may be contoured to fit individual needs
5) Tighten and secure the middle strap first, using the contact closure tab
6) Tighten and secure the two other contact-closure straps
7) Fix the thumb strap, using the contact-closure tab, then tighten and secure it.
8) Finally tighten the finger strap. The strap can be removed and adjusted as required.

ATTENTION: THE STAY CAN BE BENT ONLY ONCE, OTHERWISE THE STAY WILL BE BECOME WEAK AND BREAK.

CARE
Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION
Aluminium, Polyamide, Polyurethane, PVC, Elastane, Polyester, Thermoplastic Polyurethane

 **NOT MADE WITH NATURAL RUBBER LATEX**

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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