REVISIONS			
REV	DESCRIPTION	DATE	APPROVED
A	INITIAL RELEASE PER COR# 21199	03-11-13	TV

LABELING TYPE: IFU (PURCHASED)

STYLE: 9 PANEL

FOLD: BINDERY-ACCORDION, FOLD IN HALF

SIZE: 26.500" X 11.625"


MATERIAL: 50# OFFSET SMOOTH

PRINT: 1/1 BLACK, TOP/BOTTOM

COLOR: BLACK


PRIMARY UOM: EA

PURCHASING UOM: EA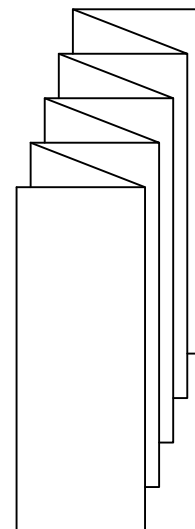

(ACCORDION FOLD)

SUGGESTED VENDOR: RR DONNELLY
VENDOR# 427106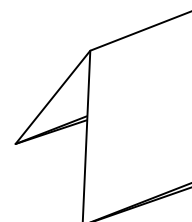

(FOLD DOWN)




2. PDF PRINT FILE ATTACHED IN CIMAGE.

1. ARTWORK LOCATED IN O:/DOC CONTROL/PACKAGING

NOTES: UNLESS OTHERWISE SPECIFIED.

SPECIFICATION CONTROL DRAWING

1. NRL () 2. DNR () 3. NO LATEX (X)		THE INFORMATION AND DESIGNS CONTAINED IN THIS DOCUMENT ARE CONFIDENTIAL. NEITHER RECEIPT NOR POSSESSION THEREOF, CONFERS OR TRANSFERS ANY RIGHT TO REPRODUCE OR DISCLOSE THE DOCUMENT, ANY PART THEREOF, ANY INFORMATION CONTAINED THEREIN, OR TO PRACTICE ANY METHOD OR PROCESS WITHIN, EXCEPT BY THE EXPRESS WRITTEN CONSENT FROM DJO.			
UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES TOLERANCES ARE: DECIMALS ANGLES .X ± - ± - .XX ± - .XXX ± -		DRAWN LATEST D. FAZENDE	DATE 02-27-13	 VISTA, CALIFORNIA 92081-8553 U.S.A.	
		DESIGN ORIGINAL D. FAZENDE	02-27-13		
		DESIGNED LATEST D. FAZENDE	02-27-13		
MATERIAL: NOTED		ENGINEER CHECK NA	NA	TITLE IFU, MINITRAX WALKING BRACE	
FINISH: NONE		PROD. RELEASE T. VANDENBERG	02-27-13		
		SIZE A	DRAWING No. 13-7068	REV. A	SCALE: NONE SHEET: 1 OF 1

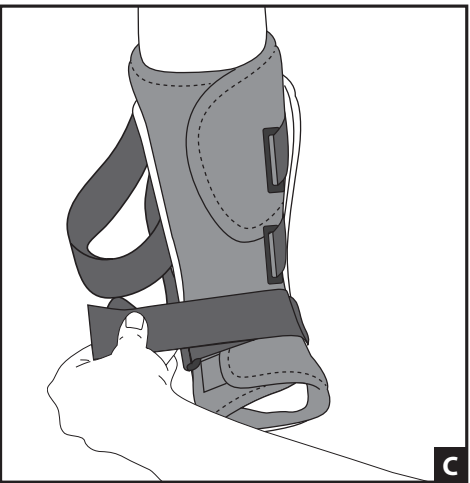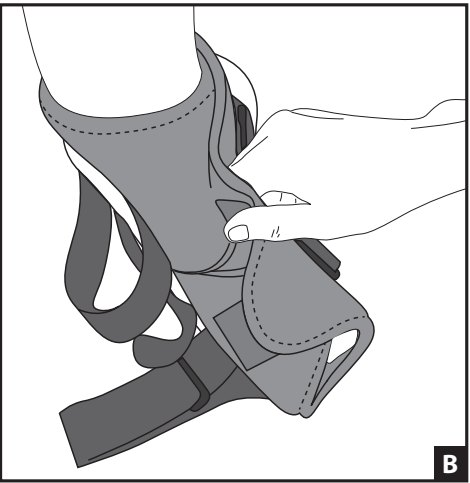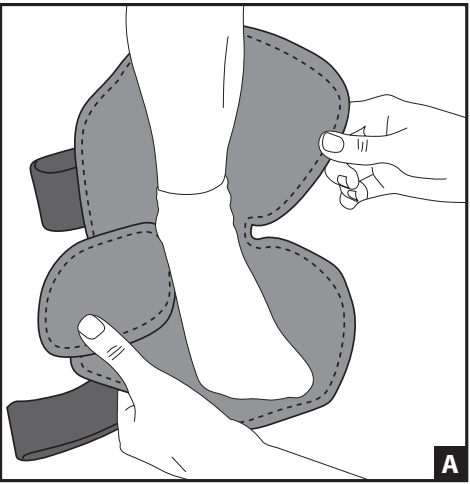ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS: The MINITRAX walking brace is effective for the treatment of forefoot, midfoot, and hindfoot injuries; foot fractures; soft tissue, tendon or ligament injuries or procedures; acute or post-operative care (ie. bunionectomy, removal of a bunion, osteotomy, removal of bone near a damaged joint, etc.); and ankle sprains. Effective in fracture healing.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS: If any pain, swelling, sensation changes, or any unusual reactions occur while using this product, consult your medical professional immediately. Do not use this device on patients incapable of communicating physical discomfort. Like all lower extremity immobilizers, such as casts or braces, patients without sensation (ie: post-op anesthesia, neuropathies, etc.) should be monitored frequently for "hot spots", skin irritation or wound management. Use caution when walking on slippery or wet surfaces to avoid injury.

APPLICATION INFORMATION:

A cotton or cast sock will enhance comfort – NOT INCLUDED

- 1) PREPARE BRACE
 - Unfasten straps and open liner.
- 2) APPLY BRACE
 - While seated, place foot in brace (Fig. A).
 - Position heel in back of brace. Wrap liner around foot and then ankle (Fig. B).
 - Secure straps from bottom to top (Fig. C). Tighten until snug and comfortable.
- 3) REMOVE BRACE
 - To remove the brace, unfasten the straps, open the liner and remove the boot.

CLEANING INSTRUCTIONS: The foam liner may be hand-washed in 86°F/30°C water with mild detergent and air-dried. Do not put in dryer or use other heat source to dry.

MATERIAL COMPONENTS:

Shell: NYLON
Straps: WOVEN NYLON
Liner: NYLON/POLYURETHANE/POLYESTER (OUTER/ FOAM/INNER MESH)
Binding: POLYESTER
Outer Sole / Insole: EVA / RUBBER

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

INTENDED FOR USE ON A SINGLE PATIENT.

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