REVISIONS			
REV	DESCRIPTION	DATE	APPROVED
C	LOGO CORR. CAD SPEC REL. COR# 15656	11-18-08	TV
D	REVISED PER COR# 18502	09-06-11	CG

LABELING TYPE: IFU (PURCHASED)

STYLE: 9 PANEL

FOLD: BINDERY-ACCORDION, FOLD IN HALF

SIZE: 26.500" X 11.625"


MATERIAL: 50# OFFSET SMOOTH

PRINT: 1/1 BLACK, TOP/BOTTOM

COLOR: BLACK


PRIMARY UOM: EA

PURCHASING UOM: EA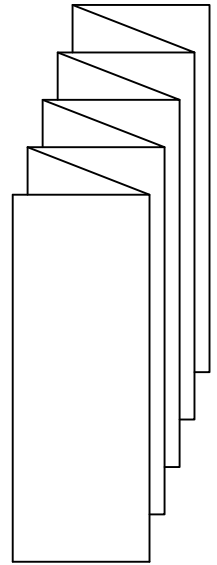

(ACCORDION FOLD)

SUGGESTED VENDOR: RR DONNELLY
VENDOR# 427106

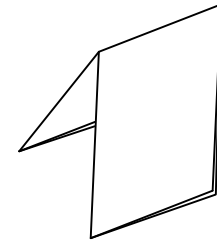(FOLD DOWN)

2. PDF PRINT FILE ATTACHED IN CIMAGE.

1. ARTWORK LOCATED IN O:/DOC CONTROL/PACKAGING

NOTES: UNLESS OTHERWISE SPECIFIED.

SPECIFICATION CONTROL DRAWING

1. NRL ()
2. DNR ()
3. NO LATEX (X)

THE INFORMATION AND DESIGNS CONTAINED IN THIS DOCUMENT ARE CONFIDENTIAL. NEITHER RECEIPT NOR POSSESSION THEREOF, CONFERS OR TRANSFERS ANY RIGHT TO REPRODUCE OR DISCLOSE THE DOCUMENT, ANY PART THEREOF, ANY INFORMATION CONTAINED THEREIN, OR TO PRACTICE ANY METHOD OR PROCESS WITHIN, EXCEPT BY THE EXPRESS WRITTEN CONSENT FROM DJO.

UNLESS OTHERWISE SPECIFIED
DIMENSIONS ARE IN INCHES
TOLERANCES ARE:

.X ± .1
.XX ± .03
.XXX ± .005

ANGLES
±1

DRAWN LATEST
D. FAZENDE

DESIGN ORIGINAL

DESIGNED LATEST
D. FAZENDE

ENGINEER CHECK
S. OCHOA

PROD. RELEASE
M. ARMES

DATE
01-15-11

01-15-11

01-15-11

3-21-06

VISTA, CALIFORNIA

92081-8553 U.S.A.

TITLE

PI, CAST SHOES/SANDALS

MATERIAL:
NOTED




FINISH:
NONE

SIZE
A

DRAWING No.
13-1322-0-00000

REV.
D

SCALE: NONE
SHEET: 1 OF 1

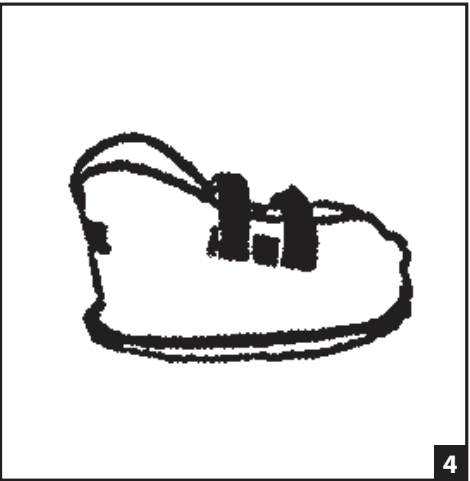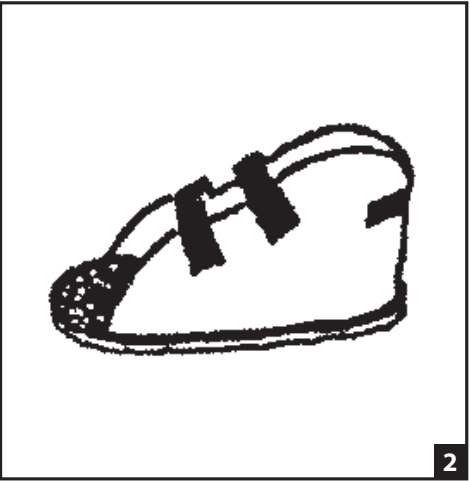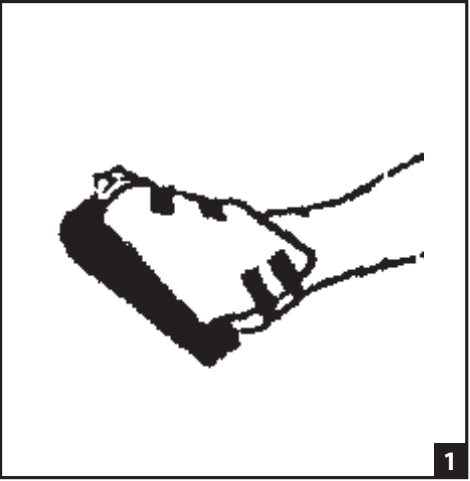ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS: To provide support and protection to the foot and cast during ambulation.

APPLICATION INSTRUCTIONS:

- A) Slip the shoe or sandal over the cast.
B) Secure the hook and loop straps for a comfortable fit.

USE AND CARE: Clean with damp cloth. Dry thoroughly before next use. Cast sandals may be hand laundered with mild detergent in warm water and air dried. The sole may be wiped clean with damp cloth.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

WARNING: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2010 DJO, LLC