ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

- The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to: Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Wrist sprains, strains and injuries.
- Pre/post-surgery wrist immobilization.

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.

APPLICATION INSTRUCTIONS

- Loosen straps and slip hand into brace.
- Place the thumb in the corresponding opening at the top of brace (left or right).

Note:

- The second opening must remain empty.
 - The hand and wrist should be gripped between the two stays (palmar and dorsal).
 - If needed, move the dorsal stay to ensure correct position.
 - Stays are malleable and may be contoured to fit individual needs.
- Tighten and secure the middle strap first, using the contact closure tab.
 - Then tighten and secure the two other contact-closure straps.

Adjust the tightness of the straps according to compression and comfort needed.
Ensure that the edge of the brace with the buckles overlaps the opposite one.

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COMPOSITION

Aluminium, Polyester, Polyamid, Polyurethane, PVC

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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