ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE
The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS
• For use following surgery or trauma to the heel & rear foot.

CONTRAINDICATIONS
• None

WARNING & PRECAUTIONS
If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INSTRUCTIONS
Unfasten or loosen closures. Slip foot into the shoe. Secure closures to a snug but comfortable fit.

COMPOSITION
• Outer shell: Polyester, Polyurethane, Nylon, Polyoxymethylene (Thermoplastic)
• Inner sole: Polyester, Polyurethane, EVA (Ethylene Vinyl Acetate)
• Outer sole: TPR (Thermoplastic polymer)

CARE
Shoe may be washed by wiping down with a damp cloth.

 **NOT MADE WITH NATURAL RUBBER LATEX**

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2017 DJO, LLC
MDSS GmbH