LEAD STYLETS / ELEKTRODEN-MANDRINS / MANDRINS DE SONDE / GUÍAS PARA ELECTRODOS / STILETTI PER ELETTRO-CATETERI / LEADSTILETTEN / STYRTRÅDAR / ΣΤΥΛΕΟΙ ΑΠΑΓΩΓΩΝ / ESTILETES PARA ELECTROCATÉTERES / LEDNINGSSTILETTER / VODICÍ DRÁTY / ELEKTROT STÍLELERİ / ELEKTRODEMANDRENG

Tapered, limber / Verjüngt, biegsam / Coniques, flexibles / Cónicas, flexibles / Rastremati, flessibili / Taps, flexibel / Avsmalnande, medelmjuk / ΟΞύληκτοι, ευλύγιστοι / Afunilados, maleáveis / Tilspidset, bøjelig / Se zúženým koncem, poddajný / Konik biçimli, esnek / Tilspisset, fleksibel
REF:

6032	45 cm (0.36 mm)
6033	52 cm (0.36 mm)
6034	58 cm (0.36 mm)

Straight, limber / Gerade, biegsam / Droits, flexibles / Rectas, flexibles / Diritti, flessibili / Recht, flexibel / Rak, medelmjuk / Ευθύγραμμοι, ευλύγιστοι / Rectos, maleáveis / Lige, bøjelig / Rovný, poddajný / Düz, esnek / Rett, fleksibel
REF:

6061	45 cm (0.36 mm)
6062	52 cm (0.36 mm)
6063	58 cm (0.36 mm)

Tapered, firm / Verjüngt, steif / Coniques, rigides / Cónicas, rígidas / Rastremati, rigidi / Taps, stug / Avsmalnande, hård / ΟΞύληκτοι, άκαμπτοι / Afunilados, rígidos / Tilspidset, stiv / Se zúženým koncem, tuhý / Konik biçimli, sert / Tilspisset, stiv
REF:

6035	45 cm (0.40 mm)
6036	52 cm (0.40 mm)
6037	58 cm (0.40 mm)

Straight, firm / Gerade, steif / Droits, rigides / Rectas, rígidas / Diritti, rigidi / Recht, stug / Rak, hårdt / Ευθύγραμμοι, άκαμπτοι / Rectos, rígidos / Lige, stiv / Rovný, tuhý / Düz, sert / Rett, stiv
REF:

6064	45 cm (0.40 mm)
6065	52 cm (0.40 mm)
6066	58 cm (0.40 mm)

J-shaped, wide radius/long reach, soft / J-förmig, großer Radius, weich / En J, large rayon, souples / Preformadas en J, radio grande, blandas / Preformati a J, a raggio ampio, morbidi / J-vormig, grote radius, zacht / J-formade, stor radie, mjuka / Σχήματος J, μεγάλης ακτίνας, εύκαμπτοι / Curva em J, de grande raio/longo alcance, flexíveis / J-formet, stor radius/lang rækkevidde, blød / Ve tvaru J s velkým ohybem/velkým dosahem, měkké / J şekilli, geniş yarıçaplı/uzun erişimli, yumuşak / Vid radius / J-formet, med lang rekkevidde, myke
REF:

6053	45 cm (0.36 mm)
6054	52 cm (0.36 mm)
6055	59 cm (0.36 mm)

J-shaped, large radius, soft / J-förmig, großer Radius, weich / En J, large rayon, souples / Preformadas en J, radio grande, blandas / Preformati a J, a raggio ampio, morbidi / J-vormig, grote radius, zacht / J-formade, stor radie, mjuka / Σχήματος J, μεγάλης ακτίνας, εύκαμπτοι / Curva em J, grande raio, flexíveis / J-formet, stor radius, blød / Ve tvaru J s velkým ohybem, měkké / J şekilli, geniş yarıçaplı, yumuşak / J-formet, stor radius, myk
REF:

6056	60 cm (0.36 mm)
------	-----------------

J-shaped, large radius, firm / J-förmig, großer Radius, steif / En J, large rayon, rigides / Preformadas en J, radio grande, rígidas / Preformati a J, a raggio ampio, rigidi / J-vormig, grote radius, stug / J-formade, stor radie, hård / Σχήματος J, μεγάλης ακτίνας, άκαμπτοι / Curva em J, grande raio, rígidos / J-formet, stor radius, stiv / Ve tvaru J s velkým ohybem, tuhé / J şekilli, geniş yarıçaplı, sert / J-formet, stor radius, stiv

REF:

6057	45 cm (0.40 mm)
6058	52 cm (0.40 mm)
6059	59 cm (0.40 mm)
6060	60 cm (0.40 mm)

ENGLISH

STERILE. Sterilized with ethylene oxide. For one procedure only. Do not resterilize. Do not use opened or damaged packages.

INTENDED USE
For use with Guidant implantable transvenous leads.

INSTRUCTIONS FOR USE
Insert a straight stylet into the lead lumen to direct lead passage and to position the lead in the appropriate heart chamber.

CAUTION: Do not curve the stylet while it is in the lead. If a curved stylet is preferred, gently curve the straight stylet before inserting in the lead.

Insert a J-shaped stylet to assist placement of the lead in the atrium.

NOTE: Partially withdraw the stylet prior to taking threshold measurements. The stylet **MUST BE REMOVED** before connecting the lead to the pulse generator.

For further instructions, see the appropriate Physican's Lead Manual

SYMBOLS ON PACKAGING

The following symbols may be used on accessory packaging and labeling (Table 1).

Symbol	Definition
	Do not reuse
	Sterilized using ethylene oxide
REF	Reference number
	Use by
	Date of manufacture
	Lot number
	Consult instructions for use
CE0086	CE mark of conformity with the identification of the notified body authorizing use of the mark

Warranty Disclaimer
Guidant disclaims all express and implied warranties for this product, including without limitation any implied warranties of merchantability or fitness for a particular purpose. Buyer assumes all risk of loss or damages arising from use of this product.

Boston Scientific Corporation acquired Guidant Corporation in April 2006. During our transition period, you may see both the Boston Scientific and Guidant names on product and patient materials. As we work through the transition, we will continue to offer doctors and their patients technologically advanced and high quality medical devices and therapies.

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All rights reserved. ML 07/08