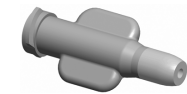
ACUITY™ X4 FLUSHING TOOL/WIRE GUIDE
ACUITY™ X4 FLUSHING TOOL/WIRE GUIDE
OUTIL DE NETTOYAGE ACUITY™ X4 / FIL
GUIDE
STRUMENTO DI LAVAGGIO/INTRODUTTORE
GUIDA ACUITY™ X4
ACUITY™ X4 FLUSHING TOOL/
DRAADGELEIDER
ACUITY™ X4 SPOLVERKTYG/LEDARGUIDE
ACUITY™ X4 SKYLLE VÆRKTØJ/WIRE
GUIDE
ACUITY™ X4 SKYLLEVERKTØY/VAIERGUIDE

REF 4604

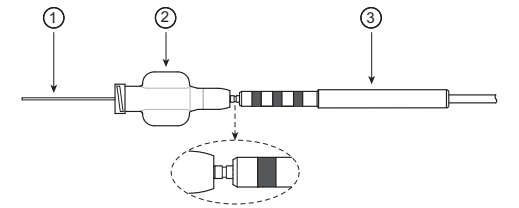
Contains one (1) ACUITY™ X4 Flushing tool/Wire guide.
Enthält ein (1) ACUITY™ X4 Flushing tool/Wire guide.
Contient un (1) outil de nettoyage ACUITY™ X4 / Fil guide.
Contiene (1) Strumento di lavaggio/Introduttore guida ACUITY™ X4.
Bevat een (1) ACUITY™ X4 Flushing tool/ draadgeleider.
Innehåller (1) ACUITY™ X4 Spolverktyg/ledarguide.
Indeholder ét (1) ACUITY™ X4 Skylle værktøj/Wire guide.
Inneholder (1) ACUITY™ X4 Skyllevertkøy/vaierguide.

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ENGLISH

This literature is intended for use by professionals trained or experienced in device implant and/or follow-up procedures.

WARNING:
For single patient use only. Do not reuse, reprocess, or resterilize. Reuse, reprocessing, or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness, or death. Reuse, reprocessing, or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness, or death of the patient.

STERILE
Sterilized with ethylene oxide. Do not use if the package is open or damaged. Return the unused device to Boston Scientific. For single use only-do not resterilize devices.

INTENDED USE
The ACUITY™ X4 Flushing tool/Wire guide is a delivery system accessory for use during the implant of an ACUITY X4 lead. It is compatible with both luer lock and luer slip tip syringes used for flushing the lead. The integrated wire guide is intended to ease insertion of a guide wire into the lumen at the terminal of the lead.

INSTRUCTIONS FOR USE

Flushing the Lead
Prior to implant, flush the selected lead with heparinized saline using the flushing tool with integrated wire guide. Insert the terminal pin of the lead into the flushing tool, and then attach a suitable syringe for flushing. The flushing tool is compatible for use with luer lock or luer slip tip syringes. Note that a portion of the terminal pin will remain visible when fully inserted into the flushing tool (see Figure 1).

Inserting the Guide Wire
A wire guide is integrated into the flushing tool to facilitate the insertion of the guide wire (see Figure 1). Use of a 0.014 inch maximum in diameter guide wire is recommended. Flush the guide wire hoop with heparinized saline prior to use and then, with the flushing tool still attached to the lead, preload the lead with the guide wire.

For further instructions, see the Boston Scientific ACUITY X4 Physician's Lead Manual and instructions accompanying the guide wire.

WARRANTY
r this product, including without limitation any implied warranties of merchantability or fitness for a particular purpose. Buyer assumes all risk of loss or damages arising from use of this product.

ACUITY is a trademark of Boston Scientific or its affiliates.

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Boston Scientific (Australia) Pty Ltd