**Label identification:** ENFit Transition Connector**Date:** 2 Marzo 2015**Pantone reference numbers:** nero**Proof:** Z**Font size and type:** Metaplus 8**Cedic code:** C00633

IFU C00633

## Enteral Distal End ENFit™ Transition Connector

Catalogue  
number

0922

Consult  
instructions for useDo not  
re-use

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**EN: Indications for Use:** This transition connector is intended for connecting an enteral giving set with an ENFit connector to an enteral catheter with funnel. **Specification:** Ready to use. Not sterile. Not made with natural rubber latex. Not formulated with DEHP or BPA. **Warnings:** Use this transition connector ONLY if an enteral catheter equipped with an ENFit connector is NOT available. Use ONLY with enteral feeding systems. NOT for intravenous use. This transition connector does not protect against inadvertent misconnection with connectors of non-enteral healthcare applications. Do NOT use if package is open or damaged. Do NOT use if labeling is incomplete or illegible. **Duration of Use:** Do NOT use for more than 48 hours. **Please note:** Do not reuse, as the connector may have been contaminated during use and become a source of infection. Follow good clinical practice and national/local guidelines for safe disposal.

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Manufacturer

Cedic s.r.l. Via Liberazione 63/9, 20068 Peschiera Borromeo (MI), Italy

ENFit is a registered trademark of GEDSA used with their permission

C 00633 - 13Feb2015

Label identification: ENFit Transition Connector

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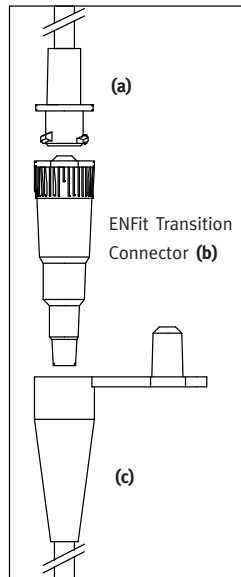
**EN: Instructions for use:** 1. Wash hands and use clean technique. 2. Remove protective cap (if applicable) from the enteral giving set distal end (a). 3. Securely screw the transition connector (b) over the enteral giving set distal end. 4. Remove any protection (if applicable) from the enteral catheter funnel (c). 5. Confirm the transition connector is being connected to an enteral catheter only. DO NOT connect to non-enteral connectors. 6. Securely fit the transition connector into the enteral catheter funnel. 7. Follow the enteral catheter instructions for use. 8. Discard the transition connector together with the enteral giving set when delivery is complete within a maximum of 48 hours. Do not disconnect the transition connector from the giving set.

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Distributed by:  
Boston Scientific Corporation  
300 Boston Scientific Way  
Marlborough, MA 01752  
USA  
USA Customer Service  
888-272-1001

**AUS** Australian Sponsor Address  
Boston Scientific (Australia)  
Pty Ltd  
PO Box 332  
BOTANY  
NSW 1455  
Australia  
Free Phone 1800 676 133  
Free Fax 1800 836 666

**ARG** Argentina Local Contact  
Para obtener información de  
contacto de Boston Scientific  
Argentina SA, por favor, acceda al  
link [www.bostonscientific.com/arg](http://www.bostonscientific.com/arg)

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