Instructions For Use

DA Brugsanvisning
DE Gebrauchsanweisung
EL Οδηγίες χρήσης
ES Instrucciones de uso
FR Mode d'emploi
IT Istruzioni per l'uso
KO 사용 지침
NL Gebruiksaanwijzing
PL Instrukcja obsługi
PT Instruções de utilização
SV Bruksanvisning
ZH 使用说明

TrapEase® | EN

EN | TrapEase® Single and Quad Chamber Polyp Traps

Intended Use:

TrapEase® polyp traps are intended for use in suction retrieval of endoscopically removed polyps.

Warnings and Precautions:

Do not re-use.

Directions for Use:

- To capture specimen, connect suction system tubing to center port of TrapEase.
- Connect outside port tube to endoscope suction port.
- For Quad Chamber Polyp Trap only:
 - Align lid arrow directly over position #1.
 - After the first tissue acquisition is complete, align lid arrow over the immediate open position for normal suctioning.
 - To capture second specimen, align lid directly over position #2. Continue this process to collect additional specimens in positions #3 and #4.
- When tissue acquisition is complete, disconnect tubing from endoscope and suction system.
- To begin specimen removal, remove lid from the polyp trap.
- Remove specimen(s) from screen shelf per customary method and place each in fixative container.
- Fixative containers and specimens are ready for transport to pathology.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

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EndoChoice, Inc.
Atlantic Systems, Ltd.