# NeoClean Cap

FR

Manuel d'instructions

EN

Instruction Manual

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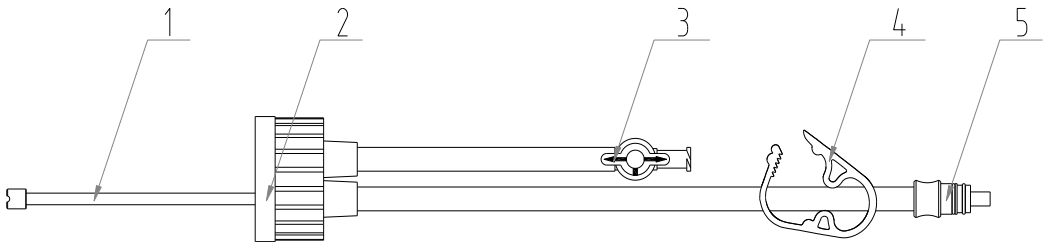
NeoClean Cap is a 24 hour use water bottle adapter that serves as an alternative to the reusable water bottle in GI Endoscopy. The NeoClean Cap design minimizes infection control risks that are associated with manual cleaning and sterilization.

## 【Intended Use】

NeoClean Cap is intended to be used with an air or CO2 and/or pump along with a sterile water source to supply air or CO2 and sterile water to an endoscope during endoscopic procedures.

## 【Types and specification】

Product sketch map as following:

Example of NeoClean Cap Pentax

1. Tube    2.Cap    3.CO2 locker    4. locker    5. Connector

## 【Warnings and Precautions】

- Guidelines for Preventing Cross Contamination:
  - **ALWAYS** prime the air/ water channel prior to inserting the endoscope into the patient.
  - **NEVER** attach or reattach the NeoClean Cap to an endoscope that is/was inside the patient and has not been reprocessed.
  - **ALWAYS** close the pinch clip before disconnecting the NeoClean Cap from the endoscope.
  - **NEVER** turn off the air processor/ CO2 while the NeoClean Cap is attached

to an endoscope.

- **DO NOT RE-STERILIZE OR REPROCESS.**

- If any of the above prevention guidelines were not followed, or there is any reason to believe that the NeoClean Cap system was contaminated, immediately discard the entire tubing and water bottle. Replace with a new tubing and unopened sterile water bottle.
- ALWAYS inspect the endoscope's o-rings and gaskets (air/ water button, endoscope connection, etc.) for damage. Damaged components can result in less than optimal performance of the NeoClean Cap.
- Sterility is not guaranteed if package has been opened or damaged. Do not use if packaging or product is damaged.

- NeoClean Cap is for 24 hours use (discard daily).

## 【Instructions for Use】

- 1) Open the sterile peel pack and remove the NeoClean Cap. Open a bottle of sterile water (250cc, 500cc, or 1000cc). Drop the weighted tube into the sterile water bottle and firmly tighten the bottle cap to ensure a secure seal. Remove

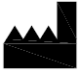
the 24hr label from the pouch and apply it to the bottle of sterile water. Write the date and time on the label.

- 2) Connect the distal tip of the NeoClean Cap to the endoscope. Align the groove on the connector and the pin on the scope side. Twist the connector to desired position; stop when undue force is required.
- 3) Place the NeoClean Cap, connected to the bottle of sterile water
- 4) NOTE: If CO2 is used, connect the stopcock of the NeoClean Cap to the CO2 tubing. Follow instructions from the CO2 manufacturer.
- 5) Turn on the air processor or CO2.
- 6) Prime the channel and test for air and water prior to insertion of endoscope. If the water pressure is low, then ensure that the water bottle is tightly closed.
- 7) If water bottle needs to be replaced, use proper aseptic technique.
- 8) When the procedure is finished: close the pinch clip and disconnect the NeoClean Cap from endoscope. Turn off the air processor/ CO2 at the conclusion of the procedure.
- 9) Discard the NeoClean Cap daily..

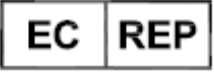
### 【Caution】

- ✓ Sterility is guaranteed unless the sterile pouch is damaged or seal is broken.
- ✓ Disposable device for single use.
- ✓ Sterilized by ETO Sterilization.
- ✓ Do NOT re-use or re-sterile.
- ✓ Check the production and expiry date printed on the sterile pouch.
- ✓ Do not store in direct sunlight, at extreme temperature, or in high humidity.
- ✓ Re-use may cause cross-infection.

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Symbol	Explanation	Symbol	Explanation
	Manufacturer		Keep away from sunlight

	<p>Authorized representative in the European Community</p>		<p>Keep dry</p>
	<p>Date of manufacture</p>		<p>Temperature limit</p>
	<p>Use-by date</p>		<p>Do not re-use</p>
	<p>Batch code</p>		<p>Consult instructions for use</p>
	<p>Sterilized using ethylene oxide</p>		<p>Caution</p>
	<p>Do not use if package is damaged</p>		
	<p>Notified Body, and product meet the basic requirements of MDD93/42/EEC</p>		