Neo Bite

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【Explication des symbols】

Symbol	Explication	Symbol	Explication
--------	-------------	--------	-------------

 **Asept InMed**
 Essentiellement simple... et tout simplement Essentiel
 Distributed by: Asept InMed SAS
 9, Avenue Mercure 31130 Quint Fonsegrives.
 Tel 05.62.57.69.00 - Fax 05.62.57.69.01
www.aseptinmed.fr

 **FARTLEY**
 Yangzhou Fartley Medical
 Instrument Technology Co., Ltd
 Beizhou Road, Lidian Town,
 Guangling District,
 Yangzhou, P.R.C., 225106
 China

 **EC REP**
 Prolinx GmbH
 Brehmstr. 56, 40239
 Düsseldorf Germany
 Tel: 0049 2131 4051968-0
 DE/0000045300

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 **FARTLEY**

Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106
China

 **EC REP**

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

【SPECIFICATIONS】

Name: Disposable Biopsy Forceps

Type: see below

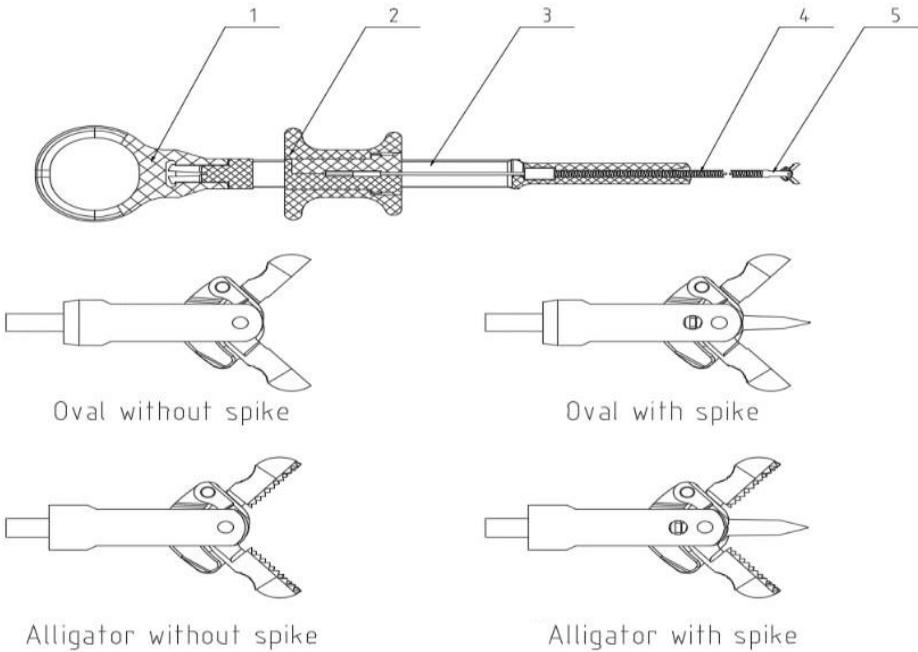
Components: Stainless Steel for Medical Use, ABS and TPU

Package: 1pcs/unit, 10pcs/case box, 100pcs/carton

Production Date: See packing

Sterilization: Sterilized by EO (ethylene oxide) gas

Validity of the sterilization: three years

1. Thumb-ring 2. Finger-ring 3. Handle 4. Sheath 5. Jaws Assembly

Art. No.	Spike	Coated	Toothed	Outer dia.(mm)	Total Length(cm)
40101	No	No	No	1,8	100
40102	No	No	No	1,8	120
40103	No	No	No	1,8	150
40104	No	No	No	2,3	160
40105	No	No	No	2,3	180
40106	No	No	No	2,3	210
40107	No	No	No	2,3	230
40108	Yes	No	No	1,8	100
40109	Yes	No	No	1,8	120
40110	Yes	No	No	1,8	150
40111	Yes	No	No	2,3	160
40112	Yes	No	No	2,3	180
40113	Yes	No	No	2,3	210
40114	Yes	No	No	2,3	230
40115	No	Yes	No	1,8	100
40116	No	Yes	No	1,8	120
40117	No	Yes	No	1,8	150
40118	No	Yes	No	2,3	160
40119	No	Yes	No	2,3	180
40120	No	Yes	No	2,3	210
40121	No	Yes	No	2,3	230
40122	Yes	Yes	No	1,8	100

Art. No.	Spike	Coated	Toothed	Outer dia.(mm)	Total Length(cm)
40123	Yes	Yes	No	1,8	120
40124	Yes	Yes	No	1,8	150
40125	Yes	Yes	No	2,3	160
40126	Yes	Yes	No	2,3	180
40127	Yes	Yes	No	2,3	210
40128	Yes	Yes	No	2,3	230
40129	No	No	Yes	1,8	100
40130	No	No	Yes	1,8	120
40131	No	No	Yes	1,8	150
40132	No	No	Yes	2,3	160
40133	No	No	Yes	2,3	180
40134	No	No	Yes	2,3	210
40135	No	No	Yes	2,3	230
40136	Yes	No	Yes	1,8	100
40137	Yes	No	Yes	1,8	120
40138	Yes	No	Yes	1,8	150
40139	Yes	No	Yes	2,3	160
40140	Yes	No	Yes	2,3	180
40141	Yes	No	Yes	2,3	210
40142	Yes	No	Yes	2,3	230
40143	No	Yes	Yes	1,8	100
40144	No	Yes	Yes	1,8	120

Art. No.	Spike	Coated	Toothed	Outer dia.(mm)	Total Length(cm)
40145	No	Yes	Yes	1,8	150
40146	No	Yes	Yes	2,3	160
40147	No	Yes	Yes	2,3	180
40148	No	Yes	Yes	2,3	210
40149	No	Yes	Yes	2,3	230
40150	Yes	Yes	Yes	1,8	100
40151	Yes	Yes	Yes	1,8	120
40152	Yes	Yes	Yes	1,8	150
40153	Yes	Yes	Yes	2,3	160
40154	Yes	Yes	Yes	2,3	180
40155	Yes	Yes	Yes	2,3	210
40156	Yes	Yes	Yes	2,3	230
40010207	No	No	No	1.8	280
40010209	Yes	No	No	1.8	280
40010210	No	No	Yes	1.8	280
40010211	Yes	No	Yes	1.8	280
40010212	No	Yes	No	1.8	280
40010213	Yes	Yes	No	1.8	280
40010214	No	Yes	Yes	1.8	280
40010215	Yes	Yes	Yes	1.8	280
40010191	No	No	No	1.0	120

【INTENDED USE】

The disposable Biopsy Forceps are used for Endoscopic mucosal tissue biopsies in the upper and lower gastrointestinal tract. The Disposable Biopsy Forceps are also used for foreign body retrieval. A thorough understanding of the

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Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106
China

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

technical principles, clinical applications, and risks associated with gastrointestinal endoscopy, mucosal biopsy and foreign body retrievals necessary before using this device. Biopsy forceps should only be used by, or under the supervision of, physicians thoroughly trained in therapeutic endoscopy, mucosal biopsy and foreign body retrieval.

【WARNINGS AND PRECAUTIONS】

Products supplied sterile are ready to use. Prior to use, check the expiry date on the package. If the expiry date has lapsed, do not use.

Do not use the product if the package is damaged or opened. All products must be stored in a dry place, away from temperature extremities.

【CONTRAINDICATIONS】

The following contraindications are cited in the medical literature with regard to polypectomy loops, papillotomes and biopsy forceps:

- No patient consent and non-cooperative patient
- Patient not on an empty stomach
- Inadequate preparation
- Fragility of the intestinal wall: eg, very red inflammation of the colon (eg, hemorrhagic rectocolitis, diverticulitis, radiation colitis, toxic megacolon)
- Peritonitis, acute abdomen, eg, intestinal perforation, ileus
- Septicemia
- Comorbidity, for example, severe cardiopulmonary disease and decompensation
- Uncontrollable hemorrhagic diatheses (less than 50,000 quick)
- Old pacemakers or pacemakers that cannot be set at a fixed frequency due to a parasystole
- Pregnancy
- Recent gastrointestinal anastomosis

As regards individual endoscopic indications for the application of the instrument, it is the responsibility of the qualified user to evaluate the contraindications and

risks associated with the application of the instrument in relation to its Benefits for each specific medical case.

Warning

The user of this device must be authorized and qualified before proceeding with the intervention in question. On the other hand, failure to follow these instructions can result in serious injury to the patient and / or user. It is essential to follow the instructions inherent in the device and its use, in particular the recommendations concerning infection control and the valid hospital protocols for the cleaning and sterilization of the reusable products possibly associated with the device. Among the most common incidents of endoscopic procedures are perforations, electric shocks, haemorrhages, infections, etc.

Failure to follow instructions may result in malfunction and / or damage to the device and / or endoscope

【DIRECTIONS FOR USE】

Visually inspect the biopsy forceps with particular attention to bends or breaks. If it appears damaged, do not use the device and contact Fartley.

Perform the diagnostic endoscopy and visualize the area to be biopsied.

Insert the biopsy forceps with the cups closed into the accessory channel in small increments.

Open the cups and advance the device into the tissue to be biopsied.

Close the biopsy forceps cups using slight pressure on the handle. Maintain gentle handle pressure to keep the cups closed while gently withdrawing the forceps from the tissue.

Withdraw the biopsy forceps from the channel. Continue to apply slight pressure on the handle to hold the cups closed.

Prepare the specimen for examination per institutional guidelines.

Dispose of for single use only products according to the guidelines for biohazardous medical waste.

【PRODUCT DISPOSAL】

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

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Symbol	Explanation	Symbol	Explanation
	Manufacturer		Keep away from sunlight
	Authorized representative in the European Community		Keep dry
	Date of manufacture		Temperature limit
	Use-by date		Do not re-use
	Batch code		Consult instructions for use
	Sterilized using ethylene oxide		Caution
	Do not use if package is damaged		Reference
	Notified Body, and product meet the basic requirements of MDD93/42/EEC		

Issued date: Oct. 2016

Rev.: A/0