Protectfilm®

Pharmaplast S.A.E.
Amria free zone 23512
Alexandria Egypt
www.pharmaplast-online.com

E C Rep Ltd,
5 FitzWilliam Square East,
Dublin 2, D02 R744, Ireland.
Tel: +353 1 2 544 944
Email: info@ecrep.ie

Description:
Protectfilm® family is a transparent protective wound dressing that consists of thin bacteria-proof, virus-proof and waterproof polyurethane film coated with hypoallergenic adhesive.
Available as **Protectfilm® Frame Style** delivery system for easier handling.
Protectfilm® Adhesive Border is a highly breathable transparent protective polyurethane film with adhesive border and adhesive free central area which prevents undesired adhesion with the wound bed and allows removal of the dressing without pain or trauma.

Intended Purpose:
• As a primary dressing, it's intended to cover and protect dry or slightly exuding wounds such as: abrasions, lacerations and areas of friction.
• As a secondary dressing, it's used as a fixation device, to secure catheter sites and to cover other primary dressings like alginates, hydrogels, hydrofibers and foams.

Precautions:
• Sterility of the dressing is guaranteed unless the pouch is damaged or opened prior to use.
• For single use only - Re-using may cause cross contamination.
• Should not be used after expiry date.
• Should not be used on patients with sensitivity to acrylic adhesive.
• Should not be used on infected or bleeding wounds unless under medical supervision.
• Repeated application may cause skin damage to patients with thin or fragile skin.

Instructions for Use:
1. Clean the wound and dry the surrounding skin thoroughly.
2. Remove the release liners of the dressing.
3. View the site through the film and central the dressing over the wound, do not stretch during application.
4. **Protectfilm® Frame Style:** slowly remove the frame while smoothing down the dressing edges, then smooth the entire dressing from the central toward the edges using firm pressure to enhance adhesion.

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Customer		Product name									
Pharmaplast		Protectfilm									
Issue date		Prepared by									
19-06-2019		Veronica Emil									
Colors		Dimension									
Blue		12 x 49 cm									
Issue Number		Revision Date		Description of Change		Rationale for Change		Amended by		Approved by	
1		N/A		New issue		N/A					
2		20-06-2019		Adding multi language		Customer needs					
3		10-06-2020		Adding language according to change to add the address items		Marketing point of view					
4		24-06-2020		Adding English translation		Marketing point of view					