PeriVac™ Kit

PERICARDIOCENTESIS KIT

English

INSTRUCTIONS FOR USE

INDICATIONS:
The Pericardiocentesis Kit is intended to withdraw fluid from the pericardial sac (membrane surrounding the heart) to evaluate the fluid for diagnostic purposes or to remove excess fluid causing compression of the heart.

CONTRAINDICATIONS:

These are absolute contraindications for pericardial aspiration, although recurrent effusion or unresolved cardiac tamponade may warrant surgical intervention.

SUMPLICATIONS/RISKS:

Complications or risks of pericardiocentesis procedures can include: catheter occlusion, infection, cardiac tamponade from puncture, atrial and ventricular arrhythmias, laceration of coronary vessels, chamber thrombus, pneumothorax, hemotrauma, and hypotension (possibly reflexogenic). Perforation of the stomach, colon, rectum, sigmoid colon, or liver is also possible.

WARNING:

Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

RISK ONLY:

This device is intended for single use only.


PRECAUTION:

Pericardiocentesis should only be performed in well-trained in the technique and aware of its risks and benefits.

DESCRIPTION OF KIT:

The Perivac Pericardiocentesis Kit is a sterile, one time kit designed to assist the clinician in draining fluid from the pericardial sac.

Pericardiocentesis procedures are ideally carried out in the operating room with fluoroscopy, ultrasound, two-dimensional echocardiographic monitoring and guidance.

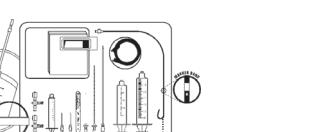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