ExBag

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ExBag

Self-opening extraction bag for minimally invasive surgery.

Caution: please read the complete user instructions before use.

STERILE - ONLY WHEN PACKAGE IS UNDAMAGED

DO NOT RESTERILISE

SINGLE USE ONLY

NOT SUITABLE FOR PROCEDURES IN WHICH LAPAROSCOPIC SURGERY IS NOT INDICATED

PURPOSE OF USE

The ExBag is used to extract tissue during laparoscopic surgery.

PRODUCT DESCRIPTION

The ExBag is a sterile instrument for single use, consisting of a preloaded extraction bag, an insertion sleeve and an extraction sleeve. It is designed for use with a standard 10 mm or 12 mm trocar.

WARNINGS AND SAFETY REQUIREMENTS

- Laparoscopic procedures must be carried out only by persons with the appropriate training and who are experienced with laparoscopic surgery.
- The ExBag is inside a protective sleeve and must not be unfolded before insertion. Unfolding the bag prematurely will damage it and the instrument will be unusable; the bag cannot be repacked.
- The ExBag must be protected against contact with sharp objects, such as cutting and crushing instruments etc..

INSTRUCTIONS FOR USE

- The insertion sleeve with the preloaded ExBag is inserted into the surgery region with the trocar.
- The pusher, which is integrated into the protective sleeve, presses the ExBag out.
- As soon as the ExBag is completely out of the insertion sleeve, the bag opens automatically.
- The insertion sleeve and the pusher are removed from the trocar and the trocar can be used for other instruments.
- Fill the bag with the resected tissue for removal.
- The filled bag is gripped at the closing wire with forceps and pulled through the trocar. This closes the opening of the bag.
- The ExBag is extracted through the trocar incision with the trocar and the forceps.

NOTE: if the contents of the bag are too large, it may be necessary to enlarge the trocar incision to make it easier to extract the bag.
- After removal the bag is emptied and disposed of.

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