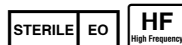
D „Hot“ Biopsiezange (Einmalgebrauch)	GB “Hot” Biopsy Forceps (Single Use)	F Pince à Biopsie „Chaude“ (Usage unique)
---	--	---

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GB Instruction Manual

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D**Warnung !**

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GB

Warning !

This instrument is for **SINGLE USE ONLY** and has to be disposed after use.
Reprocessing, resterilization and/or reuse of this product is prohibited!

Intended Purpose

The “**Hot**” Biopsy Forceps is intended for removing polyps and coagulating lesions with high frequency. Furthermore it can be used to obtain mucosal tissue samples for histological examination.

Warning !

The user of this equipment should be thoroughly trained in the applicable procedure. Furthermore, failure to read and thoroughly understand the contents of this instruction manual may result in serious injury to the patient and/or user. It is essential to follow the instructions enclosed, as well as instructions found in other instruction manuals, infection control requirements and current hospital protocols for usage, cleaning and sterilization. Possible injuries related to endoscopic procedures may include perforation, electrical burns and shock, hemorrhage, infection, explosion, etc.

! Failure to follow these instructions may result in malfunction and/or damage of the instrument or endoscope!

Storage

Please store the instrument at a temperature of 5°C to 30°C and a humidity of 30 to 60%. Keep dry and avoid exposure to sunlight and chemical fume.

General Safety Instruction

- Wearing protective garments (i.e., gloves, protective eyewear, gowns, etc.) will assist in the protection against risk of cross-contamination and injury.
- Do not use an instrument if it does not function properly. If the device has any physical defect(s), replace the instrument with a new and/or functioning instrument.
- Check the endoscope's working channel diameter prior to introducing the instrument into the endoscope. To ensure the appropriate fit of the instrument the diameter of the endoscope's working channel needs to be at least “0.2 mm” larger than the instrument's outer diameter.

! Never use instruments which do not comply with the technical specifications!

- Do not force an instrument down the working channel of the endoscope. This could result in endoscope and/or instrument damage.
- Do not under any circumstances make any modifications to the instrument or components described in this manual.
- If applicable check the active cord for cracks and/or damages on its insulation.

Inspection and Testing

- Check any damage of the package sealing
- ! *If damage or an open seal is detected, sterilization may be compromised and the instrument should be discarded and replaced.*
- Check date of sterilization.
- ! *Never use products with non-valid sterilization date.*
- Check **“Hot”** Biopsy Forceps prior to use. Form a 12 cm diameter loop and make sure, that the jaws open and close smoothly by advancing the fingerslide.
- ! *Should there occur any irregularities or damages of active cords, dispose the instruments and replace it with a new one.*
- Remove the protection cap of HF – plug at the **“Hot”** Biopsy Forceps and connect an Olympus compatible active cord to the instrument and insure the connection is tight. Connect the other end of the active cord to the electrosurgical generator and follow the manufacturer’s instructions regarding operation (cutting & coagulation mode) and proper grounding.
- ! *It is important to use an Olympus compatible plug. Other plug configurations can lead to faulty operation.*

Application

- Slowly advance the closed forceps into the endoscope’s working channel until the forceps appears at the distal end of the endoscope.
- ! *Should there be any resistance when advancing the biopsy forceps through the curvature of the endoscope, straighten the endoscope as much as possible.*
- **To remove a polyp**, grab it with the forceps by using the fingerslide and apply HF - current.
- ! *Do not touch the metal working channel outlet of the endoscope with the metallic components of a (HF) instrument while the instrument is being energized via the electrosurgical generator. Metal components of instruments such as the cups of **“Hot”** Biopsy Forceps can result in damage to the device such as short circuit, breakage and/or melting of the instrument.*
- **To take tissue out of a determined biopsy site**, grasp the mucosal by advancing the fingerslide and close the forceps cups.
- ! *Don’t use too much force when closing the forceps, they could get damaged!*

- Carefully retract the closed forceps out of the endoscope and put the polyps and tissue into a appropriate container per your institution's guidelines.

! *Make sure the forceps keeps closed while retraction.*

- If the polyp or sample is too large to close the forceps cups sufficiently, withdraw the forceps to the tip of the endoscope and then withdraw the endoscope and biopsy forceps simultaneously.
- After use, dispose the entire instrument according to legal requirements and your facility's infection control protocols.

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Endo-Therapeutics. Inc., Clearwater, Florida 33760 – USA
Advena Ltd. Hereford, HR4 9DQ – UK
Medi-Globe GmbH, Medi-Globe-Str. 1-5, 83101 Achenmühle - Germany
Medi-Globe GmbH