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GB**Warning !**

The Polyp Trap is for **SINGLE USE ONLY** and has to be disposed after use.
Reprocessing, resterilization and/or reuse of this product is prohibited!

Intended Purpose

The  **Medi-Globe** Poly Trap is used as a collection and transport device for polyps which are removed during an endoscopy for histological examination. It has four chambers to collect the polyps and four open areas for straight suction.

Warning !

The user of this equipment should be thoroughly trained in the applicable procedure. Furthermore, failure to read and thoroughly understand the contents of this instruction manual may result in serious injury of the patient and/or user. It is essential to follow the enclosed instructions, as well as instructions found in other instruction manuals, infection control requirements and current hospital protocols for usage, cleaning and sterilization. Possible injuries related to endoscopic procedures may include perforation, electrical burns and shock, hemorrhage, infection, explosion, etc.

Do not attempt to disconnect the pre-installed tubing from the trap as this may result in exposure to contents of trap during removal.

! Failure to follow these instructions may result in malfunction and/or damage of the instrument or endoscope!

Storage

Please store the instrument at a temperature of 10°C to 30°C and a humidity of 30 to 60%. Keep dry and avoid exposure to sunlight and chemical fume.

General Safety Instruction

- Wearing protective garments (i.e., gloves, protective eyewear, gowns, etc.) will assist in the protection against risk of cross-contamination and injury.
- Do not use an instrument if it does not function properly. If the device has any physical defect(s), replace the instrument with a new and/or functioning instrument.
- Do not under any circumstances make any modifications to the instrument or components described in this manual.

Inspection and Testing

- Check the Polyp Trap prior to use.
! Should any irregularities occur, dispose the instrument and replace it by a new one.

Application

- Connect tube on center port of trap to suction system (Fig. 1).
- Connect outside port to endoscope (Fig. 1).

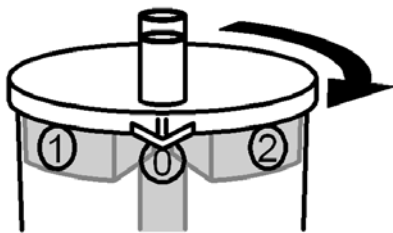
Fig. 2

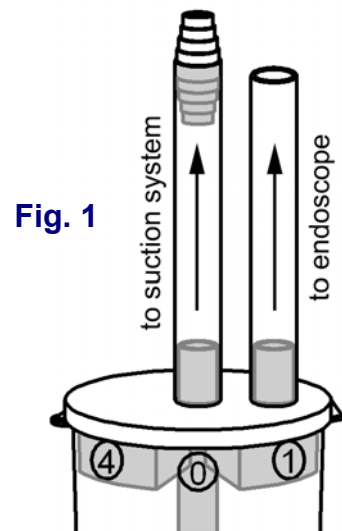
Fig. 1

- Rotate jar to align arrow on lid with any open ("0") position for normal suctioning (Fig.2).
- To entrap first polyp, rotate jar and align arrow with number "1" (Fig. 3).
- For normal suctioning after the polyp is trapped, rotate jar again to any open chamber ("0").
- Repeat the steps above for positions 2, 3 and 4 for each additional polyp you want to remove (same patient only).

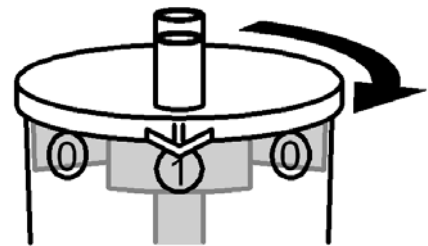
Fig. 3

- Disconnect tubing from endoscope and suction system; then remove cap from the polyp trap to remove each specimen and store in specimen jars. If you intend on using the polyp trap as a transportation device to pathology follow the instructions below.
- Bend center tube towards outside tube and insert dual-ended male adaptor to connect both tubes. Suction Polyp Trap sealed as shown lid (Fig. 4).
- Fill Suction Polyp Trap with fixative after removal of lid. Replace lid when fixative filling is complete.
- Unit is ready for transport to laboratory.

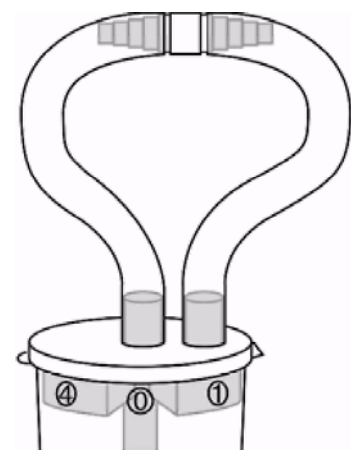
Fig. 4

- After use, dispose the Polyp Trap according to legal requirements and your facility's infection control protocols.

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Medi-Globe GmbH