# ***RotaCut***<sup>®</sup>

<b>D</b> <b>Drehbares Papillotom</b> (Einmalgebrauch)	<b>GB</b> <b>Rotatable Sphincterotome</b> (Single Use)	<b>F</b> <b>Sphinctérotome rotatif</b> (Usage unique)
---	--	---

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**GB****Warning !**

The product is for **SINGLE USE ONLY** and must be disposed of after use.  
**Reprocessing, resterilization and/or reuse of this product is prohibited!**

**Intended Purpose**

The Sphincterotome is intended to cut the papilla duodeni major and the sphincter apparatus to treat papilla stenosis.

**Warning !**

The user of this equipment should be thoroughly trained in the applicable procedure. Furthermore, failure to read and thoroughly understand the contents of this instruction manual may result in serious injury to the patient and/or user. It is essential to follow the instructions enclosed, as well as instructions found in other instruction manuals, infection control requirements and current hospital protocols for usage, cleaning and sterilization. Possible injuries related to endoscopic procedures may include perforation, electrical burns and shock, hemorrhage, infection, explosion, etc.

**! Failure to follow these instructions may result in malfunction and/or damage of the instrument or endoscope!**

**Storage**

Please store the instrument at a temperature of 5°C to 30°C and a humidity range of 30 to 60%. Keep dry and avoid exposure to sunlight and chemical fumes.

**General Safety Instruction**

- Wearing protective garments (i.e., gloves, protective eyewear, gowns, etc.) will assist in the protection against risk of cross-contamination and injury.
- Do not use an instrument if it does not function properly. If the device has any physical defect(s), replace the instrument with a new and/or functioning instrument.
- Check the endoscope's working channel diameter prior to introducing the instrument into the endoscope. To ensure the appropriate fit of the instrument the diameter of the endoscope's working channel must be a minimum of "0.2 mm" larger than the instrument's outer diameter.  
**! Never use products, which do not comply with the technical specifications!**
- Do not force an instrument down the working channel of the endoscope. This could result in endoscope and/or instrument damage.
- Do not under any circumstances make any modifications to the instrument or components described in this manual.
- If applicable check the active cord for cracks and/or damages on its insulation.

## Inspection and Testing

- Note any damage/open seal of the packaging.  
! *If damage or an open seal is detected, sterilization may be compromised and the instrument should be discarded and replaced.*
- Check date of sterilization.  
! *Never use products with a non-valid sterilization date.*
- Check the Sphincterotome prior to use. Pulling back on the finger slide of the device handle, form a bow at the distal end of the sphincterotome to insure that the instrument is operating properly. Advance the finger slide to restore the distal tip to its original position. **(1)**.  
! *Should any irregularities occur, dispose the instrument and replace it with a new one.*
- Inspect the active cord for damage.  
! *Never use an active cord which is damaged.*
- The Sphincterotome is compatible with generators of the following manufacturers:  
**ERBE, MARTIN, OLYMPUS und VALLEYLAB.**

## Application

- Carefully introduce the distal tip of the Sphincterotome into the endoscope's working channel. Continue to advance the tubing until the instrument can be visualized at the distal end of the endoscope.
  - ! *Do not force the instrument through the endoscope's working channel. If resistance is encountered, straighten the bending section of the endoscope to allow easier passage.*
  
- Advance the Sphincterotome to the desired cutting position. With  **Medi-Globe** Sphincterotome's, you may rotate the distal tip of the instrument by holding the accessory port section of the handle in a stationary position with one hand, while turning the handle in a clockwise or counter clockwise motion with the free hand.
  - ! *Make sure that the Sphincterotome is in the bile duct and not in the duct of pancreas. To ensure exact position, x-ray-technology may be required.*
  
- Connect your HF-Generator with the active cord to the connecting pin **(2)** on the finger slide of the handle **(1)**.
  - ! *The Sphincterotome can be used for cutting and coagulation.*
  - ! *Do not touch the metal working channel outlet of the endoscope with the metallic components of a (HF) device while the device is being energized via the electrosurgical generator. Metal components of instruments such as sphincterotome cutting wires can result in damage to the device such as short circuit, breakage and/or melting of the instrument.*

<b>Application</b>	<b>Maximum permissible generator power</b>	<b>Maximum permissible voltage</b>
Cutting	200 W	1300 Vpp – 1500 Vpp
Coagulating	120 W	3100 Vpp – 3200 Vpp

- By applying short HF-impulses and pushing and pulling the Fingerslide for and backwards and you can cut the papilla until the desired dept is reached.
  - ! *Please note that the distal tip must not be bowed more than 90 degrees.*
  
- Push the finger slide forward to its very distal position after the application to straighten the tip of the Sphincterotome.
  
- Carefully retract the Sphincterotome from the endoscope.
  
- After use, dispose the entire instrument according to legal requirements and your facility's infection control protocols.

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Medi-Globe GmbH