M/L-10 Multi-Fire Clip Applier Instructions For Use

Microline Surgical, Inc.
50 Dunham Road, Suite 1500
Beverly, MA 01915 U.S.A.
Tel: +1(978) 922-9810 | Fax: +1(978) 922-9209
Web: www.microlinesurgical.com
Email: info@microlinesurgical.com

09-39-30-ART Rev K 04/2017 ECO 5653

ENGLISH

M/L-10 Reusable Multifire Clip Applier - Instructions For Use

INDICATIONS FOR USE

The M/L-10 Clip Applier is indicated for occluding and ligating vessels, ducts, tracts and other tubular structures during laparoscopic and general surgical procedures

CONTRAINDICATIONS

The M/L-10 Clip Applier is not intended for use except as indicated.

COMPONENTS OF A M/L-10 REUSABLE MULTIFIRE CLIP APPLIER

The M/L-10 Multi-Fire Clip Applier consists of:

1. One reusable autoclavable handpiece
2. One removable stainless steel outer tube
3. Optional sterilization tray - CAT. #3706.
4. One sterile disposable back-loading slide cartridge, containing Titanium medium large clips (sold separately).

WARNING

If during the inspection of the shipping carton, one of the above items shows any sign of damage, do not use the instrument.

LOADING CLIP APPLIER

1. Peel the clip cartridge pouch open and remove it from the pouch.
2. Fully squeeze the handpiece trigger against the handle (Fig. 1).
3. Align the arrow on the back of the clip applier with the arrow of the cartridge as shown in Fig. 3 (arrow of the cartridge should be seen on the "non-clip" side of the cartridge).
4. Insert the cartridge all the way into the back of the handpiece (Fig. 2), making sure it is handled by its sides or proximal end behind "the ladder" in order to avoid any displacement of clips or ladder.
5. Release the trigger. (This will immediately engage a clip into the handpiece jaws).
6. The instrument is ready to be used.

FIGURE 1

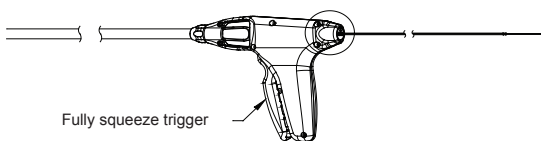
FIGURE 2


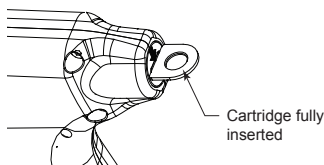
FIGURE 3

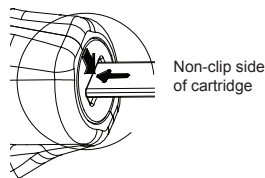OPERATION NOTE

Before firing a clip, verify that the ligation site is free of any obstructions.

1. Verify that there is a clip properly loaded in the jaws prior to inserting the clip applier through the cannula.
2. Position the clip carefully around the vessel to be ligated.
3. With full visualization of the ligation site, squeeze the trigger all the way so that the trigger touches the handle to close the clip around the vessel. The trigger will not reopen until the trigger is beyond the 7th click (see 5 below).
4. Releasing the trigger will open the clip applier jaws and automatically engage the next sequential new clip. Now the clip applier is ready for re-firing.
5. (a). For the cholangiogram procedure, whereby a clip is used to temporarily secure the catheter inside the common bile duct without occluding the catheter, the degree of clip closure can be adjusted. This is accomplished by releasing the trigger anytime after the 7th click at the ratchet mechanism. This earlier release capability allows for the desired amount of clip closure.
(b). For maximum vessel occlusion (clip fully closed) simply depress instrument trigger fully so that the trigger and handle are touching and then fully release so that the subsequent clip can be engaged in the jaws.

CAUTIONS

1. If all the clips are to be used, the last two colored clips will warn the surgeon that the cartridge is almost out of clips. The clip applier should be removed from the cavity once the last colored clip has been fired. The cartridge should then be removed and discarded.
2. Engaging the jaws over a previously applied clip should be avoided, as it may cause the handpiece to misfire.

WARNING

Make sure cartridge is removed from the handpiece after use, or prior to cleaning and sterilizing process. Microline Surgical shall not be responsible for any damage occurring as a result of a non-compliance with this warning.

DISASSEMBLING THE CLIP APPLIER HANDPIECE

To disassemble clip applier for reprocessing follow the steps below:

1. Remove and discard the disposable clip cartridge by fully squeezing the trigger against the handle as shown in Fig.4. If all of the clips have not been fired, squeezing the trigger and removing the cartridge will engage and release one clip. In this case, make sure to fire the engaged clip onto something such as a drape so that the contaminated clip does not fall onto the floor, stick to the jaws or fall inside the instrument shaft potentially causing a jam or safety hazard.
(This step serves as a warning note on the "Cleaning and Sterilization" steps. Microline Surgical shall not be responsible for any damages occurring as a result of a non-compliance with this warning.)
2. Remove the outer tube (Fig. 6) by fully squeezing the trigger and unscrewing the knob clockwise (Fig. 5).

FIGURE 4

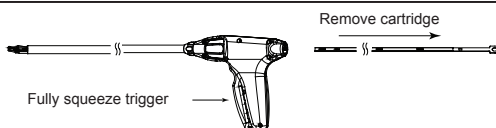
FIGURE 5


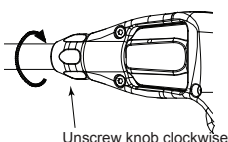
FIGURE 6

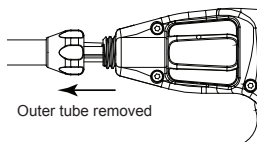NOTES

Use only Microline Surgical's sterile disposable back-loading slide cartridge. Microline Surgical will not be responsible for any damage caused by other clips.

RECOMMENDED CLEANING & STERILIZATION STEPS

WARNING

Prior to cleaning the clip applicator instrument, make sure that the disposable cartridge is removed and discarded. Microline Surgical shall not be responsible for any damages occurring as a result of a non-compliance with this warning.

NOTE

The M/L-10 clip applicator should be stored and reprocessed in a protective sterilization tray or tube.

MANUAL CLEANING

1. Immediately after surgical use, remove the shaft's outer tube. The instrument and its outer tube should be soaked in an enzymatic detergent solution.
2. After the soaking cycle, the clip applicator and its outer tube should be removed from the enzyme solution and rinsed with regular tap water.
3. The handle, shaft and rotation knob should be cleaned with a soft cloth and brush soaked in a concentrated enzymatic detergent until no further bioburden is observed. The distal end of the instrument must be cleaned with a "soft bristle" brush to avoid any damage.
4. The clip applicator handpiece and its outer tube should be rinsed in normal tap water, and then rinsed 3 times with deionized water for a final rinse. Wipe instrument with a clean soft cloth.
5. After the hospital cleaning procedure is completed, an inspection of the instrument should be done to ensure it is thoroughly cleaned.
6. It is important prior to sterilization, that a water-soluble lubricant is used to preserve the articulation and action of all movable parts of the instrument.
7. Re-assemble the outer tube on the instrument. To perform this, squeeze the handpiece trigger, mount the outer tube over the clip applicator shaft, and then secure it by screwing the tube knob counterclockwise until it is fully threaded.

NOTES

The hospital's reprocessing protocol may require that the applicator and the outer tube be reassembled after the sterilization process. Prior to wrapping the instrument for storage, it must be completely dry. Any residual moisture could result in oxidation and corrosion that could affect its performance, function and useful life.

AUTOMATED CLEANING

1. Remove the outer tube from the shaft of the M/L-10 Reusable Multifire Clip Applicator.
2. Rinse the device (s) with cold running tap water to remove gross soil and actuate any movable parts while rinsing.
3. Use a syringe to flush any lumen-like channels of the M/L-10 Reusable Multifire Clip Applicator with lukewarm tap water while fully immersed in a bath of lukewarm tap water.
4. Load the device (s) into the automated washer.
5. Ensure the following set of cycle parameters are properly programmed:
5.1 Instrument cycle HIGH

Phase	Time (minutes)	Temperature	Detergent
Pre-Wash	2:00	Cold Tap Water	N/A
Wash 1	2:00	65.5°C	Alkaline or Enzymatic Detergent
Rinse 1	00:15	Hot Tap Water	N/A
PURW Rinse	1:00	90°C	N/A
Dry	6:00	98.8°C	N/A

6. Visually inspect to ensure that all visible soil has been removed.

STERILIZATION

WARNING

Prior to sterilizing the clip applicator instrument, make sure that the cartridge is removed. Microline Surgical shall not be responsible for any damages occurring as a result of a non-compliance with this warning.

1. Prior to sterilization, the instrument must be thoroughly cleaned.
2. Insert device and outer tube into sterilization tray (Cat #3706). Wrap tray in two layers of 1-ply polypropylene wrap of appropriate basis weight to accommodate the weight of the sterilization tray and device using sequential envelope folding techniques.

Microline Surgical recommends the following validated sterilization cycles as guidelines:

Sterilization Cycle	Exposure Time	Drying Time
Gravity Cycle	15 minutes at 270°F (132°C)	30 minutes
Pre-Vacuum Cycle	4 minutes at 270°F (132°C)	30 minutes

STERILIZATION PARAMETERS FOR EUROPEAN COUNTRIES

Microline Surgical recommends the following sterilization cycle parameters for Europe, except for France and Switzerland:

Sterilization Cycle	Exposure Time at 134°C	Drying Time
Pre-Vacuum Cycle	3 minutes	30 minutes

For France and Switzerland, the following sterilization cycle parameters are recommended:

Sterilization Cycle	Exposure Time at 134°C	Drying Time
Pre-Vacuum Cycle	18 minutes	30 minutes

INSPECTION

NOTE

M/L-10 Inventory Inspection Procedure (to be performed once a month). An abbreviated inspection should be performed in the OR each time before handing the M/L-10 to the surgeon for use:

1. Check jaws for misalignment, damage and caked on bioburden.
2. Check shaft for straightness. Remove outer tube and check inner shaft for damage and caked on bio-burden.
3. Squeeze and release handle 20 plus times to check for sticking. Check handle for damage. Never pry handle and trigger open.
4. Fire a minimum of 2 clips to check clip closure, engagement into jaws and clip/jaw alignment.
5. In addition to proper and thorough cleaning (see instructions) make sure the M/L-10 is properly lubricated between cases:
 - a. Fully immerse the instrument in an approved instrument lubricant.
 - b. If unable to fully immerse the instrument make sure to thoroughly lubricate both jaw and rotation knob area.
 - c. After lubrication process, perform step 3 and 4 of the inspection if the applicator handle was sticking or locking.

ENVIRONMENTAL PROTECTION

To reduce contamination risks, the M/L-10 Clip Applicator and cartridge with unused clips shall be disposed of in compliance with all applicable local, state and federal laws and regulations.

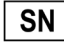
WARRANTY

Microline Surgical warrants that its instruments are free from any defects in both material and workmanship. Microline Surgical shall not be held liable for any incidental or consequential damage of any kind. Work performed on an instrument will void this warranty. Gross abuse or neglect of a Microline Surgical instrument will void this warranty.

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician.

SYMBOL DEFINITION

Following are the graphical symbols and their descriptions used in the labeling of the M/L-10 Reusable Multifire Clip Applicator:

	Do not use if package is damaged.		Catalog Number
	Consult instructions for use		Lot Number
	Manufacturer		Serial Number
	Refer to instruction manual/booklet		Non Sterile
	Federal law (USA) restricts this device to sale by or on the order of a physician.		Date of Manufacture

Not made with natural rubber latex.

Medical Device Safety Service GmbH
Schiffgraben 41
D-30175 Hannover
Germany

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