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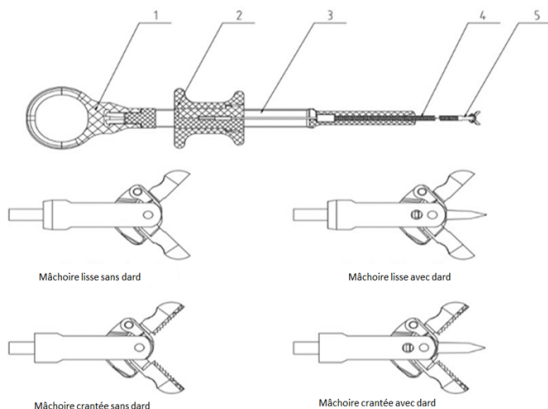
## 15. ADMINISTRATION INFORMATION

Date of modification of the leaflet: January 2020 (reference FTL-DBF-IFU-01 V/0)

Date of first affixing of the CE marking: November 2016 (CE 0197)

If you have any further questions or need information, please contact us at the following address:

Yangzhou Fartley Medical Instrument Technology Co., Ltd  
Beizhou Road, Lidian Town, Guangling District, Yangzhou,  
P.R.C. | 225106  
Tél : +86(0514)87572788 Fax : +86(0514)87268752  
[www.fartley.com](http://www.fartley.com)

### Imported by :

ASEPT INMED  
9, Avenue Mercure 31130 Quint Fonsegrives France  
Tél : +33(0)5 62 57 69 00 Fax : +33(0)5 62 57 69 01  
[www.aseptinmed.fr](http://www.aseptinmed.fr)

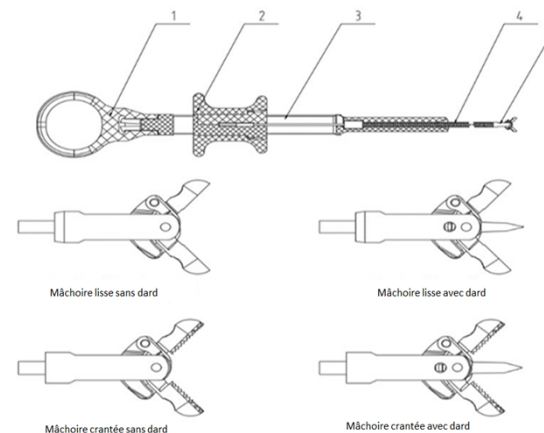
Prolinx GmbH  
Address: Brehmstr. 56, 40239, Duesseldorf, Germany  
Phone: 0049 2131 4051968-0 Fax: 0049 2131 4051968-9  
E-mail: I: info@eulinx.com DIMDI Code: DE/0000045300

EN

## NEOBITE® Biopsy Forceps

Please read this leaflet carefully before using this medical device because it contains important information.  
If you have more questions, please contact Asept InMed.

1. Thumb ring
2. Coil
3. Handle
4. coated or uncoated filament
5. Jaw

### 1. DESCRIPTION

The range is composed of the following models:

- Forceps with smooth jaw, with or without spike, coated or uncoated;
- Forceps with Toothed jaw, with or without spike, coated or uncoated.

### 2. INDICATIONS

Biopsy forceps NEOBITE® are indicated for sampling procedure under endoscope.

#### Digestive line:

- Mucosal tissue biopsy on the superior and inferior part of the digestive tube ;
- Small polyps removal from 1 to 3 mm ;
- Foreign body removal from the digestive tract;

#### Bronchial line:

- Mucosal tissue biopsy on the superior airways ;
- Foreign body removal from the superior airways ;

#### Pediatric line:

- Mucosal tissue biopsy on the superior and inferior part of the digestive tube ;
- Foreign body removal from the digestive tract;

#### Urology line:

- Mucosal tissue biopsy on the upper urinary tract ;

### 3. CONTRAINDICATIONS

Biopsy forceps are contraindicated on the patient under blood thinner treatment;  
Biopsy forceps other than the one from the pediatric line are contraindicated on child;  
Every contraindication to endoscopic procedure is also applied to the biopsy forceps;

### 4. COMPLICATION

Complication associated to all range:

- Procedural bleeding and delayed bleeding ;
- Perforation ;
- Infection ;
- Abdominal pain ;

Complication associated to the bronchial range only:

- Bronchospasm ;
- Pneumothorax/ pneumomediastinum.

## 5. WARNINGS

The act performed with the biopsy forceps must be in adequacy with the mucosa state;  
Do not use if the product is damaged, present a defect or is not functional.

## 6. USE PRECAUTIONS

Biopsy forceps must be used exclusively by physician and nurses, under the supervision of physician, properly trained for therapeutic endoscopy, for mucosal tissue biopsy, for small polyps removal, and for foreign body retrieval;  
Adopt a precise gesture to reduce the complication risk;  
Use a system to protect the mucosa (latex hood or tube) to remove a sharp foreign body.

## 7. HYGIENE PRECAUTIONS

Respect the general hygiene and asepsis rules used by the health facility during and endoscopy procedure.

## 8. FREQUENCY OF USE

Single use only.

## 9. MODALITY OF USE

Before using the biopsy forceps, perform a diagnostic endoscopy to visualize the area that need a sampling or a removal.

1. Open the primary packaging ;
2. Visually inspect the biopsy forceps to detect any abnormality (broken pieces, bent ...);
3. Activate several time the forceps handle, inserting your thumb into the dedicated ring and making the coil go back and forth, to verify the correct forceps opening and closing, corresponding to no space between the two surfaces of the jaw;
4. Insert progressively the biopsy forceps, jaw closed, inside the operator channel of the endoscope, until visualizing it at the distal end of the operator channel;
5. When the endoscope is placed close to the sampling area, approach, open then close the forceps jaw carefully by activating the handle, controlling this operation directly on the endoscope video monitor;
6. Remove the biopsy forceps from the operator channel, maintaining the jaw closed;
7. When removed from the operator channel, open the forceps and remove the sample with the curette.

## 10. STORAGE AND TRANSPORT CONDITIONS

Biopsy forceps must be conserved, stored and transported at temperature between 5°C and 35°C.

## 11. DISPOSABLE CONDITIONS

Used devices must be eliminated according to the hygiene protocol of the health facility and to the actual regulation.

## 12. MATERIAL

Biopsy forceps are manufactured with stainless steel.

## 13. PACKAGING AND STERILIZATION

Biopsy forceps are unitary packaged and sterilized with ethylene oxide;  
Cannot be re-sterilized;  
Do not use if the sterile primary packaging is damaged;  
Expiring delay: 3 years (maintain performance and sterile state);  
Do not use if expired.

## 14. SYMBOLS

	Manufacturer		Sterilized with ethylene oxide		Do not use if packaging is damaged
	Warning		Do not re-sterilize		Reference
	Special instruction for use		Single use		Batch number
	Temperature limits between 5°C and 35°C		Expiry date		Manufacturing date

## 15. ADMINISTRATION INFORMATION

Date of modification of the leaflet: January 2020 (reference FTL-DBF-IFU-01 V/0)

Date of first affixing of the CE marking: November 2016 (CE 0197)

If you have any further questions or need information, please contact us at the following address:

Yangzhou Fartley Medical Instrument Technology Co., Ltd  
Beizhou Road, Lidian Town, Guangling District, Yangzhou,  
P.R.C. | 225106  
Tél : +86(0514)87572788 Fax : +86(0514)87268752  
[www.fartley.com](http://www.fartley.com)

### Imported by :

ASEPT INMED  
9, Avenue Mercure 31130 Quint Fonsegrives France  
Tél : +33(0)5 62 57 69 00 Fax : +33(0)5 62 57 69 01  
[www.aseptinmed.fr](http://www.aseptinmed.fr)

Prolinx GmbH  
Address: Brehmstr. 56, 40239, Duesseldorf, Germany  
Phone: 0049 2131 4051968-0 Fax: 0049 2131 4051968-9  
E-mail: l: info@eulinx.com DIMDI Code: DE/0000045300