Neo Bag Grip®

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Confidentiality: this document contains information that is confidential and proprietary property of Fartley Technology. Neither this document nor the information therein may be reproduced, used or distributed to or for the benefit of any third party without prior written consent of Fartley Technology.

Imported by: Asept InMed SAS
9, Avenue Mercure 31130 Quint Fonsegrives.
Tel 05.62.57.69.00 - Fax 05.62.57.69.01
www.aseptinmed.fr

FARTLEY

Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106
China

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

【Intended Use】

Disposable Endoscopic Retrieval Bag is designed to assist in the retrieval of infectious or suspicious tissue and gall stones during endoscopic procedures.

Imported by: Asept InMed SAS
9, Avenue Mercure 31130 Quint Fonsegrives.
Tel 05.62.57.69.00 - Fax 05.62.57.69.01
www.aseptinmed.fr

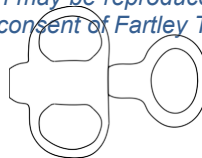
FARTLEY

Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106
China

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

Confidentiality: this document contains information that is confidential and proprietary property of Fartley Technology. Neither this document nor the information therein may be reproduced, used or distributed to or for the benefit of any third party without prior written consent of Fartley Technology.

Imported by: Asept InMed SAS
9, Avenue Mercure 31130 Quint Fonsegrives.
Tel 05.62.57.69.00 - Fax 05.62.57.69.01
www.aseptinmed.fr

FARTLEY

Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106
China

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

Confidentiality: this document contains information that is confidential and proprietary property of Fartley Technology. Neither this document nor the information therein may be reproduced, used or distributed to or for the benefit of any third party without prior written consent of Fartley Technology.

Product Code No.	Handle	Volume (mL)
40350006	Yes	200
40350007	Yes	500
40350008	Yes	800
40350009	Yes	1200
40350010	Yes	1500
40350011	No	200
40350012	No	400
40350013	No	800
40350014	No	800

Imported by: Asept InMed SAS
9, Avenue Mercure 31130 Quint Fonsegrives.
Tel 05.62.57.69.00 - Fax 05.62.57.69.01
www.aseptinmed.fr

FARTLEY

Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106
China

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

0197

Page 7 of 9

【Warnings and Precautions】

Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques. Consult the medical literature relative to techniques, complications and hazards prior to the performance of any surgical procedure.

- Use of the Endoscopic Retrieval Bag requires use of a 10mm or larger trocar sleeve.
- The Retrieval Bag is packed in a sheath and should not be deployed prior to insertion. Premature deployment of the pouch from the sheath may damage the pouch and render the device inoperative. The pouch cannot be repackaged.
- The Endoscopic Retrieval Bag must be inserted through the trocar sleeve as packaged.
- Fartley did not design this device to be reprocessed or reused, and therefore cannot verify that reprocessing can clean and/or sterilize or maintain the structural integrity of the device to ensure patient and/or user safety.
- This disposable medical device is not intended for reuse. Any institution, practitioner, or third party who reprocesses, refurbishes, remanufactures, resterilizes, and/or reuses this disposable medical device must bear full responsibility for its safety and effectiveness.

【Directions for Use】

- 1) Following recommended technique, prepare the patient for the endoscopic surgical procedure.
- 2) Using sterile technique, remove the Retrieval Bag from its package.
- 3) To insert the Retrieval Bag (DO NOT ACTIVATE HANDLE): Grasp the shaft and insert the Retrieval Pouch sheath through a 10mm or larger trocar sleeve.
- 4) Deploy and open the pouch.
- 5) Grasp tissue to be removed and place into the pouch.

Imported by: Asept InMed SAS
9, Avenue Mercure 31130 Quint Fonsegrives.
Tel 05.62.57.69.00 - Fax 05.62.57.69.01
www.aseptinmed.fr

FARTLEY
Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106
China

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

0197

- 1) Close the pouch by pulling back the handle spool. For the pouch to remain closed, tension must be maintained on the handle spool during the removal process.
- 2) Remove the pouch with tissue.

【Caution】

- ✓ Sterility is guaranteed unless the sterile pouch is damaged or seal is broken.
- ✓ Disposable device for single use.
- ✓ Sterilized by EO Sterilization.
- ✓ Do NOT re-use or re-sterile.
- ✓ Check the production and expiry date printed on the sterile pouch.
- ✓ Do not store in direct sunlight, at extreme temperature, or in high humidity.
- ✓ Re-use may cause cross-infection.

【Product Disposal】

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

A MEDI-GLOBE COMPANY

Imported by: Asept InMed SAS
9, Avenue Mercure 31130 Quint Fonsegrives.
Tel 05.62.57.69.00 - Fax 05.62.57.69.01
www.aseptinmed.fr

Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106
China

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

0197

[Symbol Explanation]

Symbol	Explanation	Symbol	Explanation
	Manufacturer		Keep away from sunlight
	Authorized representative in the European Community		Keep dry
	Date of manufacture		Temperature limit
	Use-by date		Do not re-use
	Batch code		Consult instructions for use
	Sterilized using ethylene oxide		Caution
	Do not use if package is damaged		Reference
	Notified Body, and product meet the basic requirements of MDD93/42/EEC		

File No.: FL/CE-10-02-IFU

Rev.: A/1

Issued date: 2020-08-03

A MEDI-GLOBE COMPANY

Imported by: Asept InMed SAS
9, Avenue Mercure 31130 Quint Fonsegrives.
Tel 05.62.57.69.00 - Fax 05.62.57.69.01
www.aseptinmed.fr

Yangzhou Fartley Medical Instrument Technology Co., Ltd
Beizhou Road, Lidian Town, Guangling District, Yangzhou, P.R.C., 225106 China

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300