NeoClean Cap Hybride

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Date: Juillet 2018

Revision: 0

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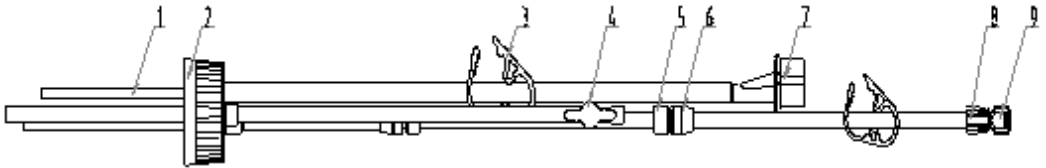
Hybrid NeoClean Cap is a 24 hour use water bottle adapter that serves as an alternative to the reusable water bottle in GI Endoscopy. The NeoClean Cap design minimizes infection control risks that are associated with manual cleaning and sterilization.

【Intended Use】

The Hybrid NeoClean Cap (tubing and accessories to accommodate various GI endoscopes and irrigation pumps) is intended to provide irrigation via sterile water and to supply air (via an air pump) or CO₂ (via a CO₂ supply) along with sterile water during GI endoscopic procedures when used in conjunction with an irrigation pump or cautery unit.

【Types and specification】

Product sketch map as following:

1. Tube 2. Cap 3. locker 4. CO₂ locker 5. Lock buckle 6. Tube connector 7. Olympus adapter 8. Check valve 9. Water jet adapter

【Warnings and Precautions】

- Guidelines for Preventing Cross Contamination:
 - **ALWAYS** prime the air/ water channel prior to inserting the endoscope into the patient.
 - **ALWAYS** prime the auxiliary water channel prior to inserting the GI endoscope into patient.
 - **NEVER** attach or reattach the Hybrid NeoClean Cap to an endoscope that is/was inside the patient and has not been reprocessed.
 - **ALWAYS** close the pinch clip before disconnecting the clean connecting

tube from the endoscope.

- **NEVER** turn off the air processor/ CO2 while the clean connecting tube is attached to an endoscope.
- **DO NOT REMOVE THE BACKFLOW VALVE.** The backflow valve is permanently attached to the irrigation tubing.
- The backflow valve attached to the Hybrid NeoClean Cap must be in place. If the backflow valve is missing, then discard the entire Hybrid NeoClean Cap, accessories and water bottle.
- **DO NOT RE-STERILIZE OR REPROCESS.**

- If any of the above prevention guidelines were not followed, or there is any reason to believe that the clean connecting tube system was contaminated, immediately discard the entire tubing and water bottle. Replace with a new tubing and unopened sterile water bottle.
- **ALWAYS** inspect the endoscope's o-rings and gaskets (air/ water button, endoscope connection, etc.) for damage. Damaged components can result in less than optimal performance of the clean connecting tube.
- Sterility is not guaranteed if package has been opened or damaged. Do not use if packaging or product is damaged.

- Hybrid NeoClean Cap is for 24 hours use (discard daily).

- **PRECAUTION:** If sterile water bottle needs replacing during a procedure, refrain from depressing the air/water valve. There is a potential for backflow to occur if the air/ water valve is depressed while no sterile water bottle is connected to the tubing. If this occurs, replace the tube immediately

【Instructions for Use】

- 1) Open the sterile peel pack and remove the clean connecting tube. Open a bottle of sterile water (250cc, 500cc, or 1000cc). Drop the weighted tube into the sterile water bottle and firmly tighten the bottle cap to ensure a secure seal. Remove the 24hr label from the pouch and apply it to the bottle of sterile water. Write the date and time on the label.
- 2) **NOTE:** If CO2 source is used, connect CO2 input to backflow valve of CO2 tubing input. Follow instructions from the CO2 manufacturer.
- 3) **NOTE:** To prevent water leakage, place clean connecting tube in pump head regardless of whether irrigation is needed.
- 4) Place the clean connecting tube, connected to the bottle of sterile water, in the water bottle holder provided on the pump
- 5) Holding both connectors in one hand, open the pump head, insert the pump tube of the clean connecting tube and close the pump head. (Note: Ensure that the pump tubing between the tubing connectors is centered, aligned, and carefully pulled taut before closing the pump head.) Take care to prevent the connectors from touching the ground..
- 6) Attach the auxiliary water jet connector on the Hybrid NeoClean Cap to the GI endoscope's auxiliary water port by engaging the threads and turning slowly until you feel resistance. (For specific instructions with regards to the connector, follow the IFU for the connector located in each shelf carton).
- 7) Connect the air/water connector of the Hybrid NeoClean Cap to the air/water

port of the GI endoscope with a firm pushing motion. If using a CO₂ source, connect CO₂ tubing directly to backflow valve on Hybrid NeoClean Cap.

- 8) Prime the tubing and the auxiliary water channel prior to use by activating the foot pedal; flush water through the entire GI endoscope.
- 9) Turn on the air processor or CO₂.
- 10) Prime the channel and test for air and water prior to insertion of endoscope. If the water pressure is low, then ensure that the water bottle is tightly closed.

WARNING:

To prevent cross-contamination, always prime the auxiliary water channel prior to inserting the GI endoscope into patient.

- 11) If water bottle needs to be replaced, use proper aseptic technique.
- 12) When the procedure is finished, close the pinch clip and disconnect the Hybrid NeoClean Cap from endoscope. Turn off the air processor/ CO₂ source at the conclusion of the procedure. Turn the pump off before disconnecting
- 13) Discard the Hybrid NeoClean Cap daily.

WARNING:

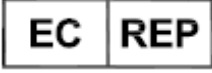
After use, these products may be potential biohazards. Handle and dispose of in accordance with accepted medical practice and applicable local, state, and federal laws and regulations.

【Caution】

- ✓ Sterility is guaranteed unless the sterile pouch is damaged or seal is broken.
- ✓ Disposable device for single use.
- ✓ Sterilized by ETO Sterilization.
- ✓ Do NOT re-use or re-sterile.
- ✓ Check the production and expiry date printed on the sterile pouch.
- ✓ Do not store in direct sunlight, at extreme temperature, or in high humidity.

✓ Re-use may cause cross-infection.

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Symbol	Explanation	Symbol	Explanation
	Manufacturer		Keep away from sunlight
	Authorized representative in the European Community		Keep dry
	Date of manufacture		Temperature limit
	Use-by date		Do not re-use
	Batch code		Consult instructions for use
	Sterilized using ethylene oxide		Caution
	Do not use if package is damaged		Notified Body, and product meet the basic requirements of MDD93/42/EEC