Adaptateur Water Jet

FR Manuel d'instructions

EN Instruction Manual

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 **FARTLEY**
Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106

 Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

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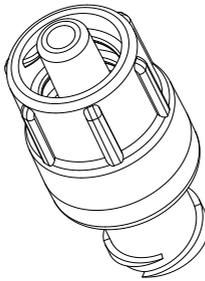
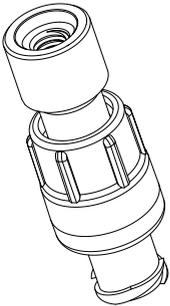
Revision: 0

【Intended Use】

The Water Jet Adapter is used in conjunction with Clean Connecting Tube (tube and accessories to accommodate various endoscopes and irrigation pumps) and is intended to provide irrigation via sterile water supply during GI endoscopic procedures when used in conjunction with an irrigation pump.

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Fujifilm

 **FARTLEY**

Yangzhou Fartley Medical
Instrument Technology Co., Ltd
Beizhou Road, Lidian Town,
Guangling District,
Yangzhou, P.R.C., 225106

Pentax

 **EC REP**

Prolinx GmbH
Brehmstr. 56, 40239
Düsseldorf Germany
Tel: 0049 2131 4051968-0
DE/0000045300

【Warnings and Precautions】

- Guidelines for Preventing Cross Contamination:
 - **NEVER** attach or reattach the disposable water Jet Adapter to an endoscope that is/was inside the patient and has not been reprocessed.
 - Endoscopic procedures should only be performed by persons having adequate training and familiarity with endoscopic techniques.
 - Consult the medical literature relative to complications, hazards, and techniques prior to the performance of any endoscopic procedure.
 - Use proper aseptic techniques to avoid contamination during device setup.
 - Exposure to bodily fluids may occur during the connection or disconnection of this device. the user is responsible for adhering to Body Substance isolation protocols.
 - Do not insert this device into the patient. the device should be attached to the auxiliary water port or biopsy channel of a flexible gastrointestinal endoscope and an irrigation source.
 - Use proper techniques to avoid risk of aspiration by the patient.
 - Always prime the irrigation port or biopsy channel prior to inserting the endoscope into the patient.

- **DO NOT RE-STERILIZE OR REPROCESS.**

- Sterility is not guaranteed if package has been opened or damaged. Do not use if packaging or product is damaged.

WARNING:

After use, this product may be a potential biohazard. handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

【Instructions for Use】

- 1) Prior to clinical use, familiarize yourself with the device.

- 2) Open the sterile pouch and attach the disposable water Jet Adapter to the Clean Connecting Tube. For use of the Clean Connecting Tube refer to its instructions for use.
- 3) Attach disposable water Jet Adapter to the water port or the biopsy channel ,then attach Clean Connecting Tube to the disposable water Jet Adapter. The disposable water Jet Adapter scope connectors are manufactured with a one-way check valve to minimize the risk of cross-contamination of the irrigation system.Connect the distal tip of the clean connecting tube to the endoscope with a firm pushing motion.
- 4) Discard the disposable water Jet Adapter after each patient use.

【Caution】

- ✓ Sterility is guaranteed unless the sterile pouch is damaged or seal is broken.
- ✓ Disposable device for single use.
- ✓ Sterilized by ETO Sterilization.
- ✓ Do NOT re-use or re-sterile.
- ✓ Check the production and expiry date printed on the sterile pouch.
- ✓ Do not store in direct sunlight, at extreme temperature, or in high humidity.

✓ Re-use may cause cross-infection.

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Symbol	Explanation	Symbol	Explanation
	Manufacturer		Keep away from sunlight
	Authorized representative in the European Community		Keep dry
	Date of manufacture		Temperature limit
	Use-by date		Do not re-use
	Batch code		Consult instructions for use
	Sterilized using ethylene oxide		Caution
	Do not use if package is damaged		Notified Body, and product meet the basic requirements of MDD93/42/EEC

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Distributed by: Asept InMed SAS