BackStop+ MiniStop+

INSTRUCTIONS FOR USE


Description:
MiniStop Plus and BackStop Plus disposal system with temporary sharps holder.

Indications and Usage:
The MiniStop Plus and BackStop Plus are intended for the collection and disposal of blood and other fluids collected during invasive procedures. The devices also include a removable temporary sharps holder for the short-term storage of needles and scalpels.

- Cautions:**
- **Rx Only:** Federal (USA) law restricts this device to sale by or on the order of a physician.
 - Contents sterile unless package is opened or damaged.
 - Single use only.
 - These devices are intended to be used in conjunction with, and not as a substitute for, normal universal blood and sharp injury precautions.
 - All needles should be flushed prior to use.
 - Contents of unopened, undamaged package are sterile and non-pyrogenic.

REUSE PRECAUTION STATEMENT
For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

- Instructions for Use:**
1. Remove the MiniStop Plus or BackStop Plus from package.
 2. Apply adhesive strip to the bottom of the basin or top of the basin lid (based on customer preference).
 3. With the adhesive strip attached, place device securely on sterile working area.
 4. Use device as indicated.
 5. If necessary, stabilize sharps holder while carefully placing or removing sharps.
 6. After use, carefully remove temporary sharps holder from basin. **Warning:** Basin may need to be stabilized with free hand.

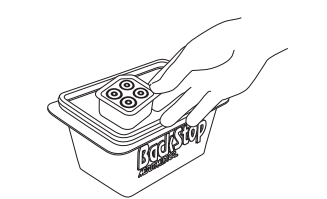7. Dispose of sharps unit according to designated guidelines for disposing of contaminated medical sharps.
8. Place lid on basin.
9. Dispose of closed waste basin according to designated guidelines for the disposal of contaminated human waste.

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Merit Medical Systems, Inc.
South Jordan, Utah 84095 U.S.A.
U.S.A. Customer Serv. 1-800-356-3748
Merit Medical Ireland Ltd,