CEN-STEP™

CENTESIS CATHETER

EN	INSTRUCTIONS FOR USE
FR	NOTICE D'UTILISATION
IT	ISTRUZIONI PER L'USO
GE	GEBRAUCHSANLEITUNG
ES	INTRUCCIONES
PT	INSTRUÇÕES DE UTILIZAÇÃO
PTBR	INSTRUÇÕES DE UTILIZAÇÃO
NL	GEBRUIKSAANWIJZING
SV	BRUKSANVISNING
DA	BRUGSANVISNING
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
TR	KULLANMA TALİMATLARI
RU	УКАЗАНИЯ ПО ПРИМЕНЕНИЮ
PL	INSTRUKCJA OBSŁUGI
CS	NÁVOD K POUŽITÍ
BG	ИНСТРУКЦИИ ЗА УПОТРЕБА
HU	HASZNÁLATI UTASÍTÁS

CEN-STEP™

CENTESIS CATHETER

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

The Cen-Step™ is a short term drainage catheter with introducer needle and vent plug.

INTENDED USE

The Cen-Step Centesis Drainage Catheter is intended for the percutaneous drainage of fluids.

CLINICAL BENEFIT

Percutaneous drainage of fluids.

CAUTIONS

- Read instructions prior to use
- **Rx Only Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician.
- Contents of unopened, undamaged package are sterile using ethylene oxide.
- Do not leave in patient longer than 24 hours.
- Store in a cool dry place
- Only persons knowledgeable of the risks involved and qualified in the procedures performed should use this device.
- Device is non-valved and leaving open to air can cause pulmonary event, take precautions to cover/close the luer end when not draining
- After use, dispose of device in a manner consistent with standard protocols for biohazard waste disposal.
- In the EU, any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the applicable Member State.

REUSE PRECAUTION STATEMENT

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or crossinfection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

INSTRUCTIONS FOR USE

Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Remove catheter and needle from protective boot.
2. Insert catheter into fluid collection site using standard insertion technique.
3. When catheter is in appropriate position remove needle from catheter.
4. Attach device with standard luer fitting to catheter hub to remove fluid.
5. Following the procedure gently withdraw the catheter and dress the site accordingly.

SYMBOL	DESIGNATION
	Use By: YYYY-MM-DD
	Lot Number
	Catalog Number
	Do Not Use if Package is Damaged
	Single Use
	Caution: Consult accompanying documents. Read instructions prior to use.
	Sterilized Using Ethylene Oxide
	Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.
	Medical Device
	Sterile Package

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Merit Medical Systems, Inc.
U.S.A. 1-801-253-1600
U.S.A. Customer Service 1-800-356-3748
Merit Medical Ireland Ltd