Espiner Tissue Retrieval System

Instructions for Use: Master E-Sac

Fannin (UK) Ltd

Yeo Bank 3, Kenn Road, Clevedon, Somerset, BS21 6TH, United Kingdom

EC REP

DCC Vital Limited

Fannin House, South County Business Park, Leopardstown, Dublin 18

T: +44 (0) 1275 878 801

E: espiner@fannin.eu


W: www.espinermedical.com

E334/7

0459

Illustration A: Master E-Sac (WG)

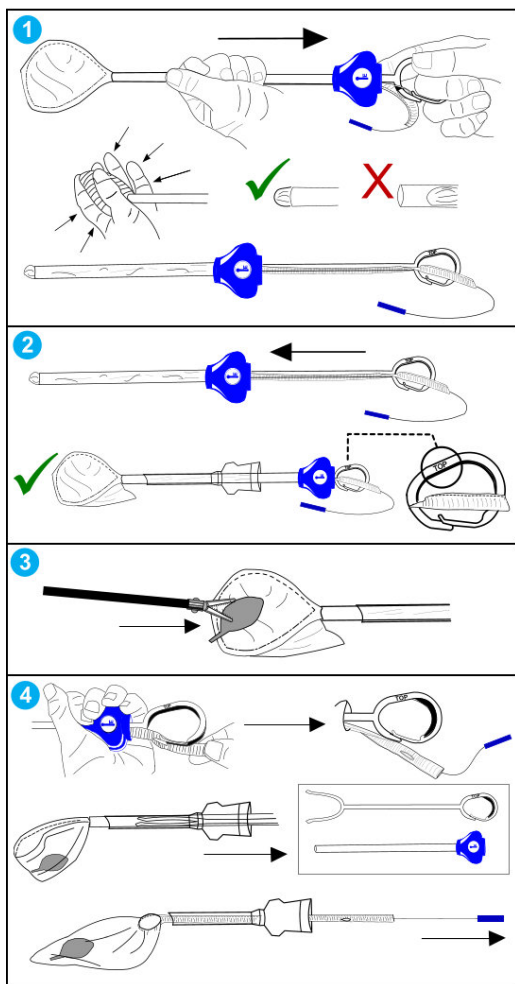	10mm ID	12mm ID
	80 - 120 WG	160 WG

ENGLISH (GB)

DEVICE DESCRIPTION:

Device Name: The Espiner Tissue Retrieval System

The Espiner Tissue Retrieval System is designed to facilitate the safe and secure removal of tissues or organs in the process of laparoscopic surgery.

They are supplied sterile (ISO 11135) with a 5 year shelf life.

INDICATIONS FOR USE:

Retrieval System consists of a family of impervious sacs, which are sterile single use devices, that can be used alone, or with a dedicated introducer system, for the capture and removal of an organ, tissue or fluid from the body cavity during laparoscopic surgery.

CONTRAINDICATIONS:

There are no Contraindications for this device.

WARNINGS & PRECAUTIONS:

The device is only to be used by surgeon who are experienced with the specific techniques required by these medical devices.

Caution: Federal (USA) and European Union law restricts the device to sale, distribution and use by or on the order of a physician.

Use the drawstring by gripping the monofilament/suture approximately 15mm from the heat shrink. The drawstring is designed for sac closure and retrieval of the sac mouth from the port site. Once outside the abdomen, continue to pull the mouth of the sac.

Caution: Do not cut, puncture, or scrape the Espiner Tissue Retrieval System.

Use tabs only for deployment of the device. Do not attempt to deploy by grasping the fabric, or by placing instruments inside the sac.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the member state.

DISPOSAL

Dispose of all used products using standard hospital practices for biohazard control.

INSTRUCTIONS FOR USE – Master E-Sac

*For Master E-Sac: Refer to Illustration A.

1. **Loading of Sac:** With the sac tail attached to the rod handle, pull the rod and sac back into the Espiner cannula. Gently squeeze the air out of the sac as it is pulled back, leaving the tip just outside the cannula.

2. **Insertion:** With the sac pulled back into the Espiner cannula, push the product through the trocar. Before pushing the rod and sac back out inside the abdomen, ensure the handle is in the correct position with 'TOP' facing up (see diagram 2).

3. **Tissue Capture:** Place the tissue inside the sac mouth. Use the sac arms and/or forceps to ease the tissue into the sac, ensuring the tissue is fully contained.

4. **Retrieval:** Once the specimen is fully contained within the sac, un-hook the sac tail from the rod handle and remove the rod completely from the sac and cannula. Pull the drawstring to close the sac mouth and secure the specimen. Remove the Espiner cannula, pull the sac tail and drawstring together, up into the trocar and ease the mouth of the sac out through the incision.

Note 1: For a larger specimen, the mouth can be re-opened outside the incision and the specimen retrieved. Once retrieved, the Espiner sac can be removed.

Note 2: When withdrawing rod, ensure it is withdrawn through the Espiner cannula and not against the trocar.

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