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【DESCRIPTION】

Disposable Grasping Forceps composed of:

- A jaw of alligator rat tooth
- A driving steel wire, coated to protect the driving wire
- An handle and a thumb ring to maintain and drive the forceps
- A finger ring to open or close the jaw
- Rotation or no rotation for easier grabbing of foreign bodies

【MATERIAL】

Materials used for Disposable Grasping Forceps are ABS,SUS304,SUS303,PE,PP,17-4PH, TPE(60165)+PP(K8003)and rubber.

It do not contain latex, phtalate, or animal derivatives.

【PACKAGING AND STERILIZATION】

The Disposable Grasping Forceps is packaged and sterilized with ethylene epoxide.

It can not be resterilized.

Validity of sterilization : 3years

【FREQUENCY OF USE】

Medical device for single use

【INTENDED USE】

This product is used in combination with endoscope to remove foreign matters.

【WARNINGS】

According to law, this instrument is intended for use by physicians who have received appreciate training.

- For single use only. Do not reuse, reprocess or resterilize. Reuse, reprocess or resterilization may compromise the structural integrity of the device and/or lead to device failure which in turn may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient. Fartley Technology assumes no liability with respect to instruments reused, reprocessed or resterilized.
- Do not use this device for any purpose other than the stated intended use.
- Visually inspect the package prior to use for any tears, irregularities or damage. If an abnormality is detected, the sterile condition of the device may have been compromised. Do not use the device.
- The operators of this product should be physicians or medical staff in the supervision of doctors who are well trained on the endoscope operation. Before use, the operators should be well known of the technical principles, clinical applications and relevant risks and countermeasures.
- Gently run your fingertips over the entire length of the catheter to check for any crushes, excessive bends, broken areas or other damage.
- Do not use this device when the package is damaged.
- This is a disposable product, thus please destroy it after using and do not reuse it.
- Do not use this product when the expiry date is exceeded.
- Do not store the package containing the device in places where it may become damaged, wet or improperly sealed. Otherwise, the sterility of the device may be compromised and pose an infection control risk or cause tissue irritation.

【CONTRAINDICATIONS】

Those specific to the primary endoscopic procedure to be performed in conjunction with the removal of polyps or foreign bodies.

【POTENTIAL COMPLICATIONS】

Those associated with gastrointestinal and urology endoscopy include, but are not limited to: perforation, hemorrhage, aspiration, fever, infection, hypotension, allergic reaction to medication, respiratory depression or arrest, cardiac arrhythmia or arrest.

【PRECAUTIONS】

Refer to the package label for minimum channel size required for this device.

- The endoscopic retrieval of foreign objects, food bolus, tissue samples, stents or excised polyps should only be performed by persons having adequate training in clinical endoscopic technique.

- Avoid blindly passing the device past any foreign object, particularly if the entire lumen is blocked.
- Use care to avoid inadvertently grasping tissue or organs not intended for retrieval.
- Keep gentle traction on the device during retrieval so that the retrieved object does not become loosened, dislodged or aspirated into the trachea.

【DIRECTIONS FOR USE】

- Insert the instrument through the endoscope channel in the view of endoscope carefully and slowly; observe if the jaw part of the grasping forceps are pushed completely out of the working channel of the endoscope.
- Open the jaw and advance the device into the foreign bodies to be removed.
- If a rotation is needed, you can revolve the rotating wheel (knob) to adjust the position of the grasping forceps (as shown in Figure 1)

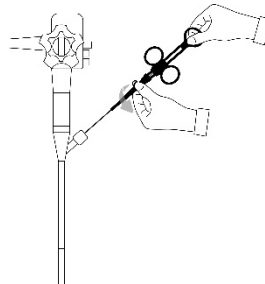
Figure 1 Rotate and adjust the Grasping forceps

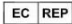
- Place the jaw around the foreign body being removed using proper endoscopic technique.
- Close the grasping forceps jaw using slight pressure on the handle. Maintain gentle handle pressure to keep the jaw closed while gently withdrawing the forceps from the foreign body.
- After the foreign body has been satisfactorily removed, withdraw the grasping forceps from the channel. Continue to apply slight pressure on the handle to hold the jaw closed.
- Prepare the specimen for examination per institutional guidelines.
- Dispose of for single use only products according to the guidelines for biohazardous Medical waste.

【NOTE】

- This product can be inserted into endoscope only when the endoscopic view is clear. Otherwise it may cause perforation, bleeding and device damage etc.
- Do not insert the device when the endoscope head is bending. For the front-end of the insert part may stretch out of endoscope front-end suddenly and thus cause perforation, bleeding and mucous membrane harm etc.
- Do not push or stretch out this device. Otherwise it may cause perforation, bleeding and mucous membrane harm etc.
- Once the endoscope and the grasping forceps have been extubated, the retrieved object, bolus, or polyp can be removed from the jaw by opening the handle forward to open the Polyp-Catcher. Specimens should be prepared according to standard technique for histologic evaluation.
- Dispose this device according to recognized hospital standard.

【STORAGE】

The single package consists of a sterile paper-plastic bag; one packaging unit consists of paper boxes and the transport package consists of paper cartons. The products should be stored in well-ventilated, dry and non-corrosive rooms.

	Manufacturer		Keep dry
	Authorized representative in the European Community		Keep away from sunlight
	Date of manufacture		Temperature limit
	Use-by date		Do not re-use
	Batch number		Instruction for use
	Sterilized using ethylene oxide		Reference
	Do not use if package is damaged		Notified Body, and product meet the basic requirements of MDD93 / 42 / EEC
	Medical device		Imported by
	Do not re-sterilized		

【PRODUCT DISPOSAL】

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

【CONTACTS】

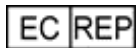
Yangzhou Fartley Medical Instrument Technology Co.,Ltd

Beizhou Road, Lidian Town, Guangling District, Yangzhou, Jiangsu, China

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Yangzhou Fartley Medizintechnik Co., Ltd
Prolinx GmbH