GB	Surgical Clipper Professional 9681
FR	Tondeuse chirurgicale professionnelle 3M™ 9681
FR	Tondeuse chirurgicale à usage professionnel 9681 3M™
DE	3M™ Professionelles OP-Rasiergerät 9681
IT	Rasoio chirurgico professionale 3M™ 9681
ES	Rasuradora quirúrgica profesional 3M™ 9681
NL	3M™ Professionele chirurgische tondeuse 9681
SE	3M™ Clipper Professional kirurgisk klippapparat 9681
DK	3M™ Professionel Kirurgisk Klipper 9681
NO	3M™ Profesjonell kirurgisk Clipper 9681
FI	3M™ Professional 9681 Kirurginen karvanleikkuri
PT	3M™ Tricotomizador Cirúrgico Profissional 9681
GR	3M™ Χειρουργική Μηχανή Κοπής Τριχών Επαγγελματική 9681
PL	Profesjonalna strzygarka chirurgiczna 9681 firmy 3M™
RU	3M™ Хирургический Клиппер Профессиональный 9681
TR	3M™ Profesyonel Cerrahi Tıraş Sistemi 9681

Non-Sterile

Product Description

The 3M™ Surgical Clipper Professional 9681 is a cordless, rechargeable, battery-operated clipper which is comprised of a clipper body, drop-in charger stand with cord (REF9682*), and single-use blade assembly (REF9680). *9683, 9684, 9685, and 9686-Additional Drop-in Chargers with Cords

Indications for Use

The 3M™ Surgical Clipper Professional 9681 is intended to remove wet or dry body and head hair from patients in preparation for any medical procedure requiring hair removal. It is intended for use in a hospital, clinic, ambulatory surgery center or other medical facilities. It is not intended for home use.

Explanation of signal word consequences

WARNING: Indicates a hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION: Indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.

WARNING to reduce the risks associated with fire and explosion, which if not avoided could result in serious injury or death and/or property damage:

- Do not operate clipper where aerosol (spray) products, flammable anesthetic mixture with air, nitrous oxide or oxygen administering equipment other than nasal or mask types are being used.
- Emergency Medical Technicians (EMTs), do not use clipper on a patient whose clothing is saturated with flammable solutions such as gasoline, fuel oil, solvents, etc.
- Avoid using around flammable alcohol or similar cleaning solution. If alcohol is used to clean the clipper, assure that the clipper and surrounding area is ventilated of fumes before plugging in the drop-in charger with cord or switching on the clipper.

WARNING to reduce the risks associated with hazardous voltage, which if not avoided could result in death or serious injury:

- Do not attempt to plug or unplug the drop-in charger with cord from an electrical outlet with wet hands.
- Unplug the drop-in charger cord before cleaning.
- Do not place or store a drop-in charger with cord where it can fall or be pulled into a tub or sink of water.
- Do not retrieve a drop-in charger with cord that has fallen into or is submerged in water or other solutions. Unplug cord from the wall outlet immediately.
- Never charge the clipper if the drop-in charger has a damaged cord or plug; if it is not working properly, if it has been dropped or damaged, or if the drop-in charger has been dropped into water.
- Do not put in fire or mutilate the battery pack during disposal as it may burst or release toxic materials.
- Do not short circuit the battery pack by bridging the battery contacts with a metal or conductive part as it may cause burns.
- Do not attempt to replace the battery pack. This could result in the risk of fire or electric shock.
- Do not charge the clipper within 1.5 meters (4.9 feet) of the patient.
- Do not charge near water.

WARNING to reduce the risks associated with cross contamination, which if not avoided could result in death or serious injury:

- Do not reuse the clipper blade assembly. Single patient use only.
- Always clean the clipper body between patients.

CAUTION to reduce the risks associated with fire, and hazardous voltage, which if not avoided may result in minor or moderate injury and/or property damage:

- Do not sterilize the clipper body, drop-in charger with cord, or blade assembly.
- Only use attachments recommended by the manufacturer.
- Always store the clipper and drop-in charger with cord in a moisture-free area.
- Do not attempt to replace the battery in the clipper.
- Do not tamper with the device's internal components. There are no user serviceable parts inside this device.
- Do not charge the clipper outdoors.
- Keep drop-in charger with cord away from heated surfaces.
- See rating plate on drop-in charger cord unit for proper wall outlet voltage.
- This device may cause (EMI) Electromagnetic Interference. If this occurs, separate product from affected device.
- The use of accessories other than those specified may result in increased emissions or decreased immunity of the clipper.
- Do not submerge the clipper in water or other solution deeper than 6 inches or longer than 15 minutes.
- Do not dismantle the clipper during the useful life of the clipper for this can affect the watertight construction of the clipper.

CAUTION to reduce the risk associated with environmental contamination, which if not avoided may result in minor or moderate injury:

- At the end of product life, dispose of all components in accordance with your governmental regulations.
- Dispose of batteries properly in accordance with your local, state, and/or government requirements.
- Do not tamper with the device's internal components. There are no user-serviceable parts inside this device.

CAUTION to reduce the risks associated with skin nicks and lacerations, which if not avoided may result in minor or moderate injury:

- Do not use the clipper with a damaged or broken clipper blade assembly.
- Do not clip in a raking manner as this technique may damage the patient's skin.

CAUTION to reduce the risk of burns, which if not avoided may result in minor or moderate injury:

- Blade temperature can reach upwards of 50°C (122°F) after prolonged use.
- Do not leave blade in the same location on skin for longer than one second.
- If using in temperatures >30°C to 35°C use a 30-minute-on followed by 20-minute-off duty cycle.

User Responsibility

Only health care professionals or other appropriately trained individuals should use this equipment.

Health care professional should follow the “To Use the Clipper” portion of this document. Additional training resources are available including: instructional online videos, a Clipper Skills Evaluation form, and in-person clinical support. Contact 3M for further information.

Monitor product to verify normal operation in the configuration in which it will be used; not tested to safety for use in all mobile emergency medical service (EMS) environments.

Warranties/3M Liability

Limited Warranty

This warranty is in lieu of any other warranty express or implied except as indicated below: This 3M™ Surgical Clipper Professional is warranted against any defects in material or workmanship for a period of 2 years from the date of purchase. If this surgical clipper is determined to be defective within the warranty period, it will be replaced without charge by 3M. In the U.S., contact your local 3M Sales Representative or the 3M Health Care Helpline – 1 800 228.3957. Helpline personnel are available Monday through Friday, from 7:00 AM to 6:00 PM Central Time. After business hours, a voice prompt system will take your product complaint information. International customers, please contact your local 3M Subsidiary for necessary shipping labels and packing list. Please include a description of the defect with the returned unit.

This warranty does not cover damage caused by misuse, negligence, accident, abuse or alteration, modification of the units by anyone other than 3M or affixation of any accessory not manufactured by 3M, or use of the surgical clipper with current or voltage other than that specified in this instruction insert. Replacements after the warranty period will be made and charged to the customer on the basis of rates which are available upon request.

EXCEPT FOR PERSONAL INJURY, 3M IS NOT LIABLE IN EITHER TORT OR CONTRACT FOR ANY LOSS OR DAMAGE, DIRECT, CONSEQUENTIAL, OR INCIDENTAL ARISING OUT OF THE USE, MISUSE, OR INABILITY TO USE THIS PRODUCT. All implied warranties are limited in duration to 2 years from date of original purchase.

This warranty does not apply to Australia and New Zealand. Customers should rely on their statutory rights.

Directions for Use

To Attach a Disposable Blade to the Clipper Body:

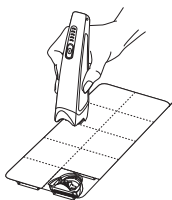
Make sure that the ON/OFF button is in the OFF position. After removing the lid of the clipper blade assembly blister pack and without touching the blade assembly, hold the clipper body over the clipper blade assembly (a). Slide the new clipper blade assembly into position until it clicks (b).

To Use the Clipper:


The patient's skin should be clean. To turn the clipper on, simply push the ON/OFF button above the LED battery gauge on the clipper to the ON position. The power indicator will light up to the corresponding battery capacity (c).

LED Battery Gauge Operation

The top three LED light gauges will be green and the bottom two will be yellow. These gauges indicate battery capacity. See (d) below. NOTE: When battery level is low, the lowest LED will blink indicating re-charge is needed. The LEDs will also blink when in the charge stand indicating that the clipper is charging. The level of charge can be determined by the light that is blinking at that time.

a.

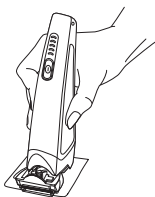b.

c.


d.


Percent of Charge

GREEN	80-100%
GREEN	60-80%
GREEN	40-60%
YELLOW	20-40%
YELLOW	0-20%

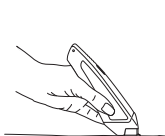
Use short, gentle strokes, holding the clipper blade assembly at a flat to slight angle to the patient's skin (e). For best results, clip against the direction of the hair growth. For contoured areas, stretch the skin taut while clipping. There are several ways to grasp the clipper. Determine the most comfortable technique that allows you to keep the clipper blade assembly at a flat to a slight angle to the skin. This technique can be performed in a push or pull motion. Never rake the clipper over the skin (f).

e.

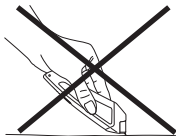f.

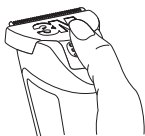g.

To Change Disposable Blade:

Push the ON/OFF button into the OFF position. Hold the clipper body in one hand. Position the clipper over an appropriate waste receptacle with the blade facing downward. With your thumb, push the blade release button to push the blade off the clipper body into waste receptacle (g).

NOTE: To reduce cross contamination between patients, remove the clipper blade assembly and discard appropriately after each use.

Care and Cleaning of the Clipper Body:

The housing of the clipper body is made of durable plastic and is water resistant. It may be wiped off with a damp cloth containing a small amount of alcohol, disinfectant, or a mild detergent solution. It can also be rinsed under running water or submersed in a high level disinfectant solution. Approved chemicals for use in cleaning include 70% alcohol, 2% bleach, Alkaline Gluteraldehyde, Quaternary ammonium chloride, or Chlor-clean solution. Allow to dry completely before placing clipper back into drop-in charger or reusing.

Do not sterilize the clipper body or blade assembly.

Charging the 9681 Battery-Operated Clipper:

To charge the clipper:

1. Push the ON/OFF button to the OFF position.
2. Clean the clipper body before returning clipper to the drop-in charger, making sure the clipper and charger contacts are clean and free of hair or other contaminants.
3. Connect the cord of the drop-in charger to an appropriate electrical outlet, making sure the cord is not pinched or kinked.
4. To be certain that the 9681 clipper is fully charged, plug it in for 3 hours before its initial use and every use thereafter.
5. We recommend leaving the clipper body in the drop-in charger in between uses. It cannot be overcharged.
 - Tested to be charged by standard facility AC outlet and conventional ambulance AC outlet (i.e. pure and modified sine wave 120/240 VAC).

Storage/ Shelf Life/Disposal

Storage and transport conditions: -20°C to 45°C, <75% RH, 86 to 106kPa

Operating conditions: 30% to 75% RH, 70 to 106kPa

0°C to 29°C: no duty cycle

>30°C to 35°C: using a 30-minute-on followed by 20-minute-off duty cycle

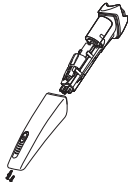
9681 Battery Pack Removal

The clipper body contains recyclable Lithium-Ion (Li-Ion) batteries, which are not intended to be replaced. When your rechargeable clipper reaches the end of its useful life, the battery pack must be removed from the clipper and be recycled or disposed of properly in accordance with your local, state, and/or government requirements. The disposal of Li-ion batteries as municipal solid waste is prohibited.

The procedure below describes the removal of the battery pack for the purpose of proper disposal. Discard the clipper after the battery pack has been removed.

h.


i.

1. Make sure that the ON/OFF button is in the OFF position.
2. Remove disposable clipper blade assembly.
3. Dry clipper and hands completely.
4. Remove the three housing screws (h).
5. Pull or pry the top and bottom housing apart from the top (i).
6. Pull motor and battery pack out.
7. Pull Li-ion battery out of the assembly.
8. Recycle battery pack or dispose of appropriately.

How supplied

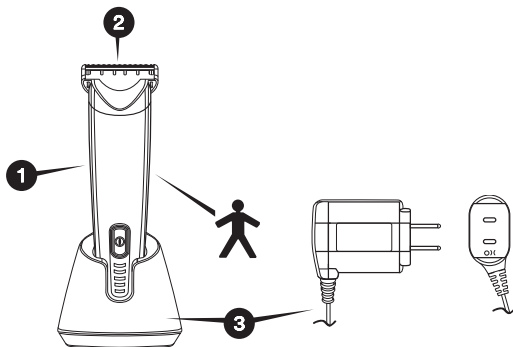Components/Accessories

1. 9681 3M™ Surgical Clipper Professional
2. 9680 3M™ Clipper Blade Assembly
3. 9682 3M™ Drop-in Charger Stand with cord (Plug Type A)*
9683 3M™ Drop-in Charger Stand with cord (Plug Type C)
9684 3M™ Drop-in Charger Stand with cord (Plug Type G)
9685 3M™ Drop-in-Charger Stand with cord (Plug Type I)
9686 3M™ Drop-in-Charger Stand with cord (Plug Type I, Argentina only)

*Not CE Marked

Note: The 9680 3M™ Clipper Blade Assembly is the only acceptable blade to be used on the 9681 3M™ Surgical Clipper Professional.

9681 Clipper Specifications

Motor Voltage:	3.6 volt DC
Current without load:	1.0 amps max.
Current with full load:	1.3 amps max.
RPM:	5500-6500
Power Adapters Charge Circuits:	All power adapters/charge circuits are 100-240 VAC, 50-60Hz.
Charging Time:	4 hours
Battery Operating Time:	120 minutes
Battery:	Lithium-Ion (Li-ion)
Clipper Weight (with blade):	180 grams (6.7 oz)
Clipper Dimensions (with blade):	15.5 cm (6.1 in.) Length 4.3 cm (1.69 in.) Diameter

THE DEVICE HAS BEEN TESTED IN ACCORDANCE WITH IEC 60601-1 EDITION 3.1 (MEDICAL – GENERAL REQUIREMENTS FOR BASIC SAFETY AND ESSENTIAL PERFORMANCE) AND IEC-60601-1-2 EDITIONS 3 AND 4 (COLLATERAL STANDARD-ELECTROMAGNETIC DISTURBANCES)

This device complies with the requirements of European Medical Device Directive 93/42/EEC.

ELECTROMAGNETIC ENVIRONMENT GUIDANCE

ELECTROMAGNETIC EMISSION

The 3M™ Surgical Clipper Professional, 9681 is intended for use in the electromagnetic environment specified below. The customer or user of the 3M™ Surgical Clipper Professional, 9681 should ensure that it is used in such an environment.		
Emissions Test	Compliance Level	Electromagnetic Environment – Guidance
RF Emissions CISPR 11	Group 1	Model 9681 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class B	3M™ Surgical Clipper Professional, Model 9681 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonics IEC 61000-3-2	Class A	
Flicker IEC 61000-3-3	N/A	

ELECTROMAGNETIC ENVIRONMENT GUIDANCE

INTERFERENCE IMMUNITY

The 3M™ Surgical Clipper Professional, 9681 is intended for use in the electromagnetic environment specified below. The customer or user of the 3M™ Surgical Clipper Professional, 9681 should ensure that it is used in such an environment.

Immunity test	IEC 60601-1-2 Test level	Compliance level	Electromagnetic environment guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ± 15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/ burst IEC 61000-4-4	± 2 kV for supply lines ± 1 kV for input/output lines	± 2 kV for supply lines Not applicable for input/output lines	
Surge IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	± 1 kV differential mode ± 2 kV common mode	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial magnetic field or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply lines IEC 61000-4-11	< 5 % U_T (>95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycle 70 % U_T (30 % dip in U_T) for 25 cycle < 5 % U_T (>95 % dip in U_T) for 5 sec	Not applicable	

Guidance and Manufacturer's Declaration – Emissions

The 3M™ Surgical Clipper Professional, 9681 is intended for use in the electromagnetic environment specified below. The customer or user of the 3M™ Surgical Clipper Professional, 9681 should ensure that it is used in such an environment.

ELECTROMAGNETIC ENVIRONMENT GUIDANCE

INTERFERENCE IMMUNITY

Immunity test	IEC 60601-1-2 Test Level	Compliance level	Electromagnetic environment guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	Not applicable	Portable and mobile RF communications equipment should be used no closer to any part of the Model 9681, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2 \sqrt{P}$ 150 kHz to 80 MHz $d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.3 \sqrt{P}$ 800 MHz to 2.7 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^ashould be less than the compliance level in each frequency range,^b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.7 GHz	3 V/m 80 MHz to 2.7 GHz	

Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Model 9681 is used exceeds the applicable RF compliance level above, the Model 9681 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Model 9681.

Recommended Separations Distances for the 3M™ Surgical Clipper Professional, 9681

The 3M™ Surgical Clipper Professional, 9681 is intended for use in the electromagnetic environment in which radiated disturbances are controlled. The customer or user of the 3M™ Surgical Clipper Professional, 9681 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF Communications Equipment and the 3M™ Surgical Clipper Professional, 9681 as recommended below, according to the maximum output power of the communications equipment.

Recommended Separation Distances Between Portable and Mobile RF Communications Equipment and 3M™ Surgical Clipper Professional, Model 9681.			
Rated maximum output power of transmitter, P [W]	Separation distance according to frequency of transmitters, d [m]		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2.7 GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

For further information, please contact your local 3M representative or contact us at 3M.com and select your country.

Explanation of Symbols:

Caution – See Instructions For Use

Equipment Button “ON”,
Equipment Button “OFF”

Type B Applied Part

Fail-safe Transformer

Direct Current

Non-Sterile

Temperature Limitation

Recycle in accordance with
your local, state, and/or
government requirements

California Energy Commission

Level VI Efficiency Standard for
external power supplies compliant

Humidity limitation

Consult Instructions for Use

Class II Equipment

Use Indoors

Splash Proof (power adapter
models 9683, 9684, 9685)

Do Not Reuse

Keep Dry

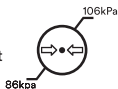
The disposal of Li-ion batteries as
municipal solid waste is prohibited

Switching mode power supply

Level V Efficiency Standard for
external power supplies compliant

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© 2018, 3M. All rights reserved.
3M is a trademark of 3M.
St. Paul, MN 55144 U.S.A.
3M Deutschland GmbH